

VEDOECO TECHNICAL INFORMATION

Definition

- Gallium thermometer Vedoeco.

General features

- Clinical thermometer in glass.
- Gallium thermometer

Applications

- For checking temperature.
- For general domestic and hospital use.
- Pack with a magnifier for an easier reading

Medical device according to the directive CEE 93/42

Modified by the directive 2007/47/CE

Implementation with D.L. n°37 del 25-01-2010

Class I measurement

Conformity

- Directive 93/42
- EN 12470-1:2000

Technical specifications



SPECIFICATIONS	VALUES
Dimension	Lenght 128 mm
Tip	Lenght 3 cm
Scale	°C
Graduation	Extension of graduation: Min 35,1 °C / Max 42 °C Subdivision of graduation: 0,1 °C
Temperature	35,1 °C - 42,00 °C
Scale resolution	Acceptable tolerance: +0,1/ - 0.15 °C
Device of maximum	Yes



VEDOECO TECHNICAL INFORMATION

Sterile	No
Store	Store in a cool, dry place, away from heat and direct sunlight
Warranty	2 years

Product Cleaning & Disinfection

- The thermometer can be cleaned with a disinfectant solution, without any limits.

Disposal

- The liquid of the thermometer is completely no- toxic.

Indications for use

- Ensure hands are clean and dry.
- Place the thermometer.
- For an oral measurement: Refrain from drinking hot or cold beverages in the 15 minutes before the measurement place the tip of the thermometer in the lower part of your mouth under the tongue.
- Under the armpit: place the thermometer with the bulb in the hollow of the armpit in contact with the skin, then lower your arm so that it supports the thermometer without exerting too much pressure.
- Rectal measurement: insert the tip of the thermometer for not more than half a centimetre into the rectum. In case of any resistance stop immediately. During the measurement maintain the thermometer in position.



VEDOECO TECHNICAL INFORMATION

How to use the thermometer

- Before taking the measurement make sure that the gallium is under the first numbered step.
- Place the thermometer in the favourite zone.
- After about 4 minutes the measurement is taken.
- After the measurement the liquid must be put under the first numbered step by shaking it.

Packaging

Labels and/or packages have the following:

Trade name
Instructions and indications for use
Manufacturer
Lot number indicating production month/year
Item bar code
Storage instruction

Packaging material

- Plastic case and pack in carton.

Manufacturer

- Geratherm Medical AG – Fahrenheitstrasse 1 D- 98716 Geschwende

Supplier

- Artsana S.p.A. Via Saldarini Catelli 1, 22070 Grandate CO ITALY

Rev 01 del 18/11/2010



VEDOECO TECHNICAL INFORMATION

Assortment

CODE	DESCRIPTION
00 023015 010 100	Gallium thermometer Vedoeco

The Technical Department

ARTSANA S.p.A.
Dr. MAURO CASSANI
Medical Devices Department
Technical Director

4/4 Artsana S.p.A. considers the information in this document confidential and non-disclosable. Artsana reserves all rights to change this document based on technical developments and amendments to relative product standards.

Rev 01 del 18/11/2015

