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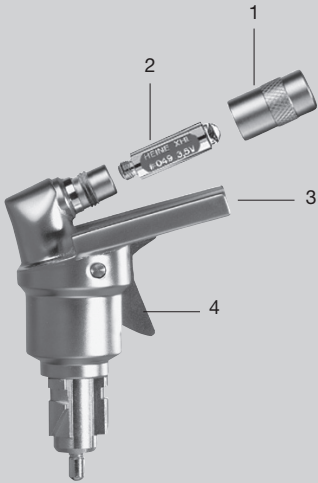
Instructions

HEINE tongue-depressor with blade ejector

Read these instructions carefully before using the HEINE tongue-depressor with blade ejector attachment and keep them in a safe place for future reference.

Safety information

Range of applications: The HEINE tongue-depressor is used to illuminate the mouth and throat with or without a tongue-blade fitted to it. Only HEINE disposable tongue blades may be used.

Safety in use: Use only handles which are specifically designed for use in medical applications. Avoid contact with mucosa or damaged skin with the tongue-depressor.

Inserting and ejecting blades

Push a HEINE disposable tongue-blade into the guide aperture (3) until the catch latches into the rectangular notch in the blade.

The blade can be ejected after use by squeezing the trigger (4).

Changing the bulb

Unscrew the bulb cover (1) from the head and remove the bulb (2). Insert a new bulb and retighten the cover (1).

Please note: The performance of this instrument can only be guaranteed if genuine HEINE bulbs are used.

Cleaning and sterilisation

The tongue-blade holder can be gas sterilised, although cleaning with alcohol is normally satisfactory. Autoclaving is not allowed.

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General Conditions of Warranty

Instead of the statutory warranty time period of 2 years, HEINE will grant a guarantee of 5 years from the date of the consignment of the goods ex works, concerning its equipment (excluding disposables, e.g. bulbs, single-use articles, and rechargeable batteries).

The guarantee covers irreproachable workmanship, on condition of the proper use of the equipment and the observation of the operating instructions. During the warranty- and guarantee time period, errors and deficiencies arising on the equipment will be rectified free of charge, in so far as such are evidenced by defective materials, processing and/or constructional errors. Should buyer complain of a material deficiency during the warranty time period, then the onus of proof is always to be on the orderer, that the product was defective already upon receipt of the goods. The statutory warranty and the guarantee do not apply to loss or damage caused by wear and tear, negligent use, the non-employment of original HEINE components and/or spares (in particular bulbs, as these have been especially developed for HEINE instruments in accordance with the following criteria: colour temperature, useful service life, safety, optical quality and performance. The statutory warranty and the guarantee do not apply to interventions by persons not authorised by HEINE or when the operating instructions are not observed by the customer. Any modification of a HEINE product with parts or additional parts which do not conform to the original HEINE specification will invalidate the warranty for the correct function of the product and further invalidate any warranty claims which result from such a change or modification. Further claims, in particular claims for replacement of loss or damage, which are experienced otherwise than directly on the HEINE product itself, are hereby excluded.

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