Deluxe Magnifier with LEDs

VM966M

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Deluxe Magnifier with LEDs from Aidapt. The Deluxe Magnifier with LEDs is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The deluxe handheld Magnifying Glass with 6 high output LED lights for reading, hobbies, crafts, maps, etc., is ideal for travel or situations where the light is subdued and features 4 x magnification.

Requires 2 x AAA Batteries (not included).

INSTRUCTIONS FOR USE

- Press and slide open the battery compartment at the base of the handle and insert the batteries following the direction shown in the compartment. Close the battery compartment lid.
- To use the LEDs and magnifier, locate the switch on the handle and press to turn on the lights. Press the switch again to turn off the lights and preserve battery life when not in use.
- Use your Deluxe Magnifier with LEDs as you would a standard magnifier, with or without the LEDs in operation.

CLEANING

Clean your Deluxe Magnifier with LEDs using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Do not submerge the Magnifier.

After cleaning, use a soft, dry cloth to wipe the magnifier and frame.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)