bioleader®

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FMBIO®

blood sampling chairs

Manuel d'utilisation
User manual

LABELIANS
Groupe CML-ID

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Installation of the chair

The chair must be installed on a flat uncluttered surface to ensure the stability of the seat during its use. After unboxing and mounting the armrests in their clamps the chair is ready for use, without need for any other assembly.

Provide enough space to the front and the back of the seat to be able to lower the back and raise the leg-rest to the horizontal position.

Provide enough space around the seat for the model with rotation, to be able to rotate the seat during its use.

Installation of the kits

Mounting of the paper roll holder (option)

- On the back of the backrest, partially unscrew the two screws above the gasspring attachment.
- Position the paper roll holder against the backrest by inserting the screws in the holes of the paper roll holder.
- Slide the paper roll holder laterally so the screwheads hold the paper roll holder.
- Screw the screws back in to lock the paper roll holder.

Mounting of the clamps and stirrups (option)

- Remove the plastic protective caps from the 2 holes located on either side of the chair seat, towards the leg-rest.
- Attach a clamp on either side of the chair seat, by inserting the 2 screws of each clamp in the holes and tightening them.

Do not force the screws: if they do not screw in completely, remove the clamp and reposition it, so as not to damage the screw thread.

- Insert the rods of the stirrups in the clamp, after having unscrewed the tightening knob beforehand.
- Position the stirrup at the desired height and tighten the tightening knob to lock the stirrup in place.

Do not rotate, lower or raise the stirrups without loosening the tightening knobs; always loosen the tightening knob before moving the stirrup so as not to damage the screw of the tightening knob.

Mounting of the tray kit (option)

The tray kit can be mounted on the vertical rods of the armrests or the vertical rods of the stirrups.

- Slide the clamp of the kit on the rod of the armrest or stirrup and position it at the desired height before tightening the tightening knob.
- Insert the vertical rod of the tray support in the hole of the clamp mounted on the armrest or stirrup, after having unscrewed the tightening knob beforehand.
- Insert the stainless-steel tray in the tray support.
- Tighten the tightening knob to lock the rod of the support.

Do not force the tightening knobs, so as not to damage the screw thread.

Instructions for use

The chair allows **blood samples** to be taken in the best safety conditions. If the chair is equipped with clamps and stirrups, it is also possible to take **gynaecologic samples**.

This chair is to be used by a **medical professional (doctor, nursing staff) within a care facility**; it must not be used in a domestic environment.

The patient or the persons accompanying him or her are not allowed to manipulate the controls of the chair.

Precautions for use

To facilitate access in and out of the chair, the armrests can be rotated to the side after having untightened the tightening knobs beforehand.

It is highly recommended **not to put any weight on the armrests** when sitting down or standing up from the chair, this might damage the tightening knobs.

Warning: **never try to rotate the armrest when the locking handle is locked**; the armrest might get unscrewed from the ball joint support. If this happens, with the locking handle locked screw the armrest back on tight.

It is **forbidden to sit down on the raised leg-rest or on the lowered backrest**, this might damage the chair.

It is forbidden to move the chair with a patient seated in the chair (for the version equipped with castors).

Maximum load capacity: **135kg**

Labelians cannot be held responsible for:

- Any use of the bioleader® blood sampling chairs other than those claimed.
- Any damage to the chair or injuries to the operator or to the patient caused by installation, use or maintenance procedures that are not conform to those described in this manual, nor from improper handling of the chair.
- Any modification made to the chair during or after installation, other than the substitution of removable components or accessories for after sales support or upgrade reasons.


Adjustment of the armrests

Untighten the handle on the side of the seat to unlock the rod of the armrest.

Adjust the height and rotation of the armrest by sliding the stainless-steel rod up or down and by turning it left or right.


Tighten the handle on the side to lock the rod of the armrest in the chosen position.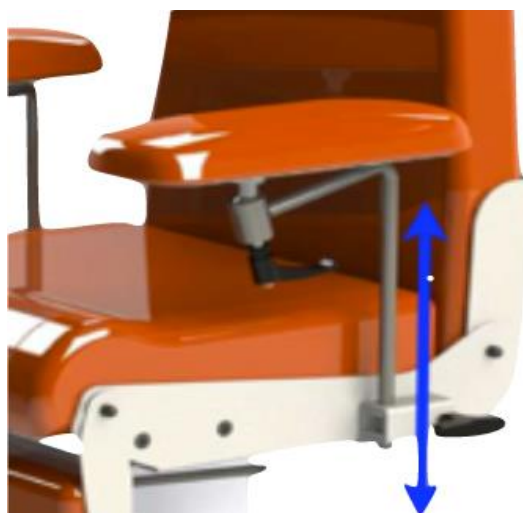

Unlock the armrest by untightening the locking handle underneath it.

Move the armrest in the desired position with a single movement.


Lock the armrest into the desired position, by tightening the locking handle.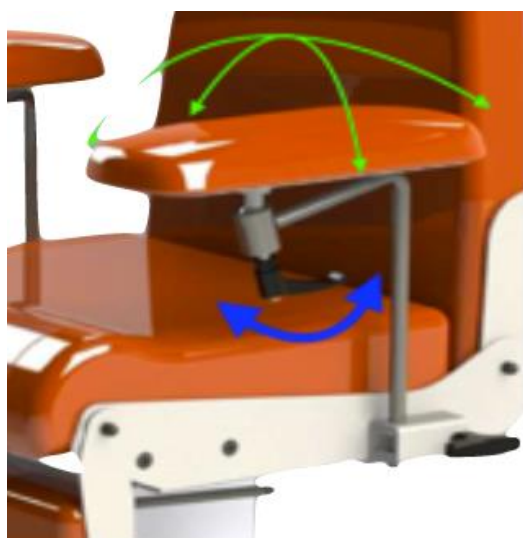

Warning: never try to rotate the armrest when the locking handle is locked (see the precautions for use).


Adjustment of the backrest

Before unlocking the backrest, place a hand on the top of the backrest to restrain it from abruptly moving forward.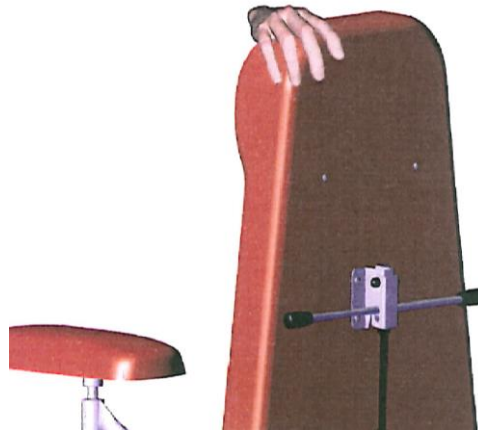


Pull the transversal lever at the back of the backrest downward to unlock the backrest; keep the lever depressed.

Adjust the angle by following the movement of the backrest with your hand – when raising and lowering it.

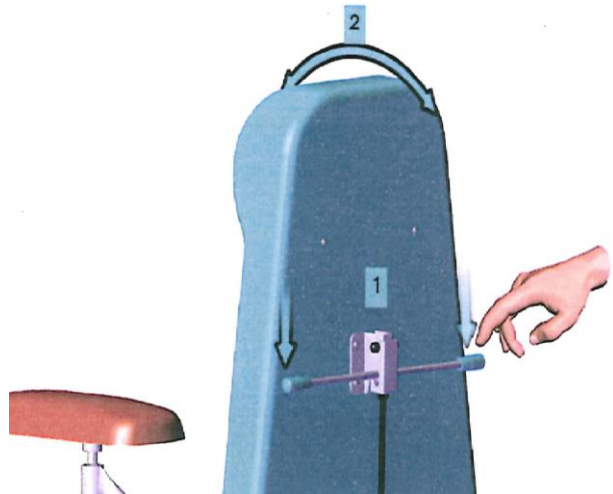
Release the transversal lever; the backrest will remain locked in the chosen position.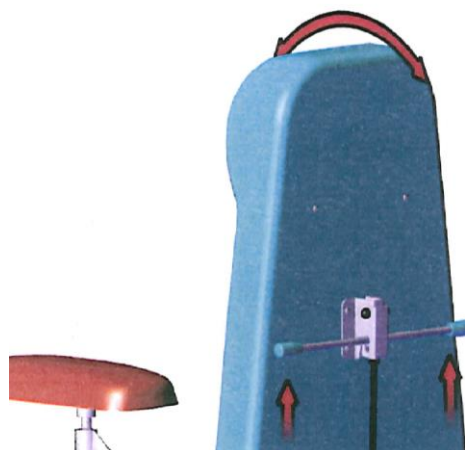


Adjustment of the leg-rest

The positioning of the leg-rest is made by simply pulling it upward; the leg-rest will lock in the desired position without requiring any other handling.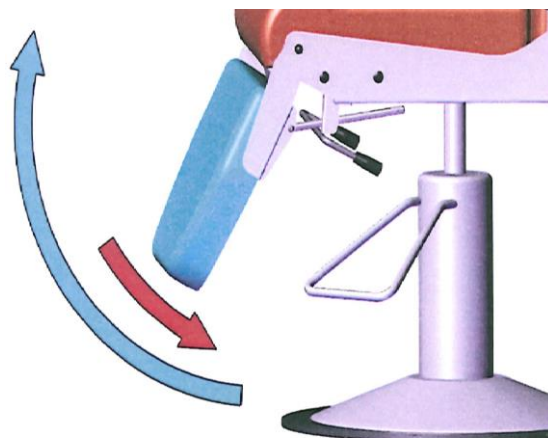


To lower the leg-rest, restrain it with one hand while pushing the transversal lever downward; position the leg-rest in the desired position.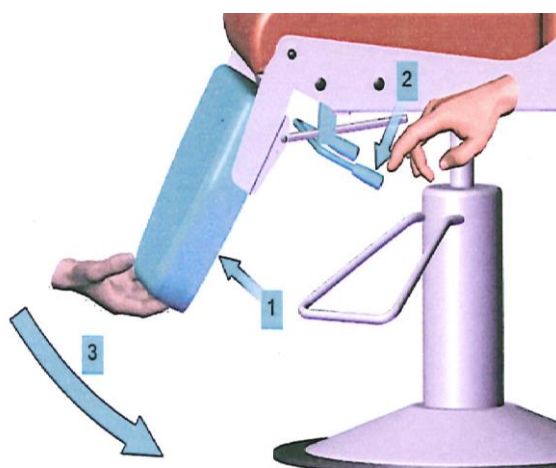

Rotation and height adjustment of the chair

Only on the FMBIO®5 model

1. **Rotation:** the chair can be rotated 360° without effort.
2. **Locking rotation:** raise the pedal with your foot to prevent rotation of the chair.
3. **Raising:** pump the pedal with your foot to raise the chair.
4. **Lowering:** completely depress the pedal to lower the chair.

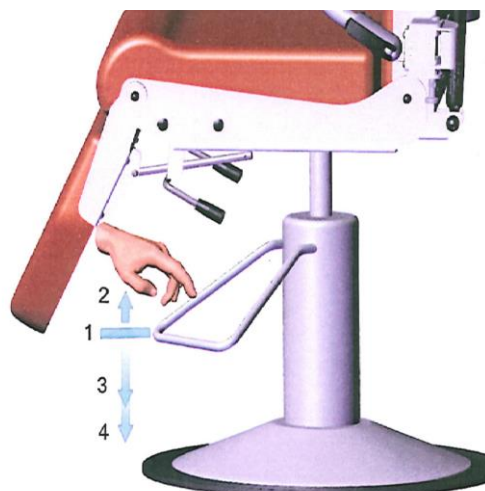Cleaning and maintenance

The upholstery is made from fire-resistant Boltaflex® imitation leather with Prefixx® coating, on Bultex® HR35 foam with shape memory. The metal structure is made from steel with an epoxy finish.

Cleaning

The upholstery and metal structure must regularly be cleaned using a **sponge and soapy water**.

Persistent stains on the upholstery can be removed with **spirit F (essence F, not white spirit)**.


It is highly recommended **not** to use other types of cleaning products such as solvents (gasoline, cleaning alcohol, acetone ...), detergents, bleach or abrasive cleaning products, to avoid damaging the upholstery.

Regular maintenance

It is required to at least **once a year**:

- Verify the different articulated parts and make sure there is no excessive movement
- Verify the screws are tightened properly
- Verify that the metal structure has not been distorted

To avoid lockage or squeaking of the moving parts, it is recommended to **regularly grease these parts** with thick vegetable grease to ensure their proper functioning – refer to the diagram on the next page for more details.

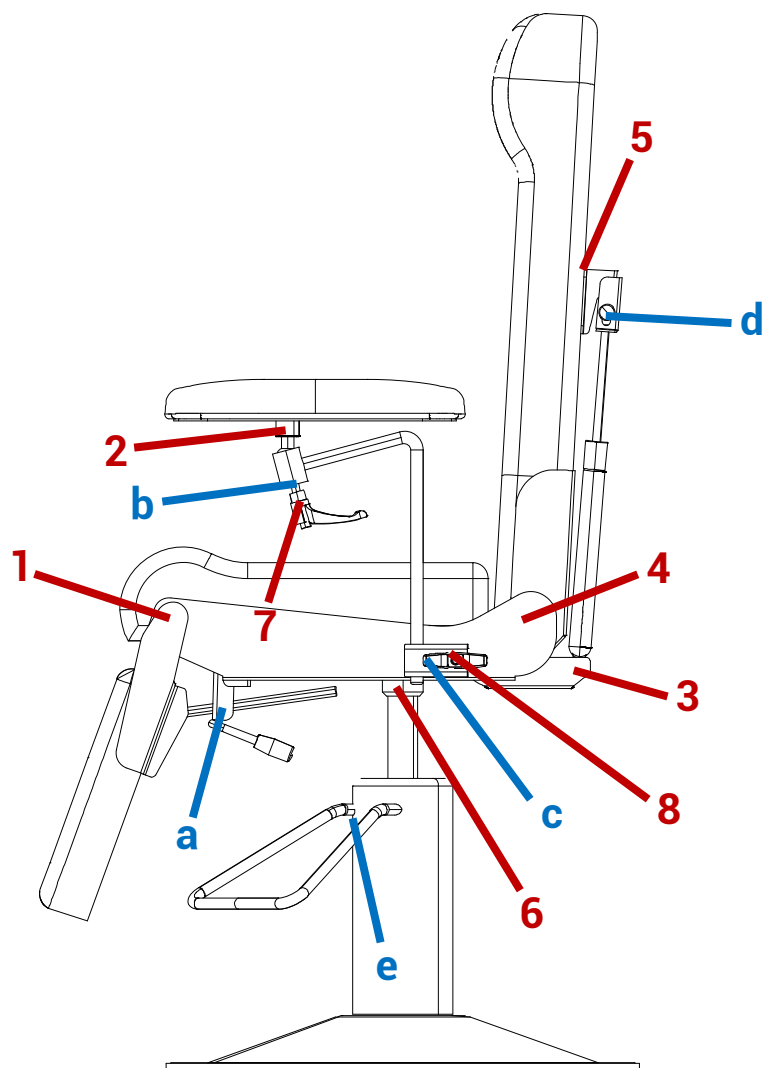Carry out the following tasks on both sides of the chair:

- Grease the axles and tightening knobs at points: 1, 4, 7, 8
- Verify and tighten if needed the fixings at points: 1, 3, 4, 5, 6
- Tighten if needed the little screw under the armrest (requires an Allen key): 2
- Verify that the tightening knobs, the clamps and the mechanisms are not distorted (rods and screw threads) at points: a, b, c, d, e
- Verify that the upholstery is not punctured or thorn, this can prevent a proper cleaning of the upholstery.

In the event of the deterioration or fracture of a component, please contact your retailer or the Labelians after-sales service to obtain a spare part or plan a repair.

After-sales service

The after-sales service must be carried out by **authorized qualified technicians**.

Please contact your retailer, your Labelians salesperson or the Labelians after-sales service in case of problems.

Accessories for your chair

Labelians offers a range of accessories to extend the capabilities of your chair:

Protective PVC covers

BMBIO10 Headrest cover (1pc)

BMBIO12 Armrest cover (1pc)

BMBIO14 Total armrest cover (1pc)

BMBIO13 Seat cover (1pc)


BMBIO11 Leg-rest cover (1pc)

Accessories

PIF

Removable tray kit (1pc)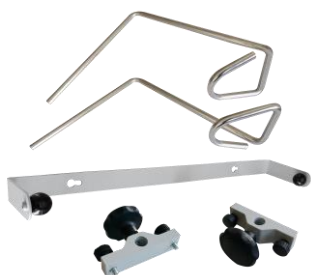

BMBIO9

Gynaecology kit containing:

1 paper roll holder
2 clamps for stirrups
2 stainless-steel stirrups


ART-06578

Paper roll holder, width 500mm

ART-06577

Clamps for stirrups 850, 2pcs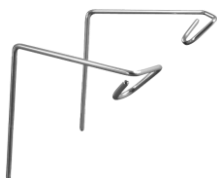

850

Stainless-steel stirrups,
diam 16mm, 2pcs

Please contact your retailer or Labelians salesperson for any information or to place your orders.

Page vide

Empty page

LABELIANS

1 rue des Palis, 77140 NEMOURS, FRANCE

Tél : +33 (0)1 64 45 42 97 - +33 (0)1 64 45 42 07

Fax : +33 (0)1 64 78 14 69

Email : bioleader@labelians.fr

www.labelians.fr

www.bioleader.fr