NORME

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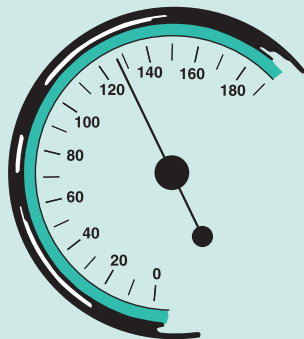
Instruction Manual for Sphygmomanometers

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NORM

This FB sphygmomanometer fulfils the requirements of the EC directive 93/42/EEC (Medical Device Directive) and is designed according to the European standard EN 1060, non-invasive sphygmomanometers part 1 and part 2. **CE 0123**. It is a product class IM (measuring function), rule 1.

INSTRUCTION FOR BLOOD PRESSURE MEASUREMENT

- Release air completely off your cuff and place the cuff at the left forearm on a level with your heart.
- Close the air release valve completely which is located on the left side of the manometer.
- Inflate the cuff with the bulb.
- Open the air release valve slowly; the speed of air release should be approx 2-3 mmHg/sec. Read the systolic and diastolic values from the scale.
- After the measurement deflate the cuff completely while opening the air release screw.

TECHNICAL DATA

Type: aneroid sphygmomanometer complete with bulb and cuff (1- or 2 tubes)
Measurement range: 0-300 mmHg
Accuracy: +/- 3mmHg
Operating Temperature: +10° C bis +40° C
Storage Temperature: -20° C bis +70° C
Graduation: 2 mmHg
mmHG: millimeter / mercurial

CLEANING

Manometer: use a soft, lightly moistend cloth. Do not use petroleum spirits, thinners or similar solvents
Cuff: after tearing off the bladder you can wash the bladder till 60° C.

TECHNICAL CONTROL OF ACCURACY

A technical check up of the high quality manometer should be done after at least 2 years. Therefore please contact a dealer who is authorized by FB or send your sphygmomanometer directly to FB-Medizintechnik, Bisingen.

DATE OF PRODUCTION

The date of production is stamped on the scale of the manometer which gives you an information about date of production and serial number (the first 2 numbers = year of production and the third number = quarter of the year).

GARANTEE

Friedrich Bosch GmbH & Co. KG guarantees for the intermentioned unit for 2 years period beginning with the date of its sale through the dealer. This guarantee covers all defects coming from faulty material or manufacturing - rubber parts and electronic parts excepted - but not from inexpert use. This card to be filled up from the dealer has to be returned with the defect instrument. Without guarantee card there is no claim on our guarantee service within the scope of the above mentioned regulations.