□	Babywaage Gebrauchsanweisung.....	2
GB	Baby scale Instruction manual.....	4
F	Pèse-bébé Mode d'emploi	6
E	Báscula para bebés Manual de instrucciones.....	8
I	Bilancia pesaneonati Istruzioni per l'uso	10

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ENGLISH

Dear customer,

thank you for choosing a product of our range. Our name stands for high and thoroughly checked quality-products in the field of warmth, gentle therapy, diagnosis/blood pressure, massage and air. Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With regards

Your Beurer-Team

! Important Notes

1. Caution

- Protect your personal scales from impact with hard objects, moisture, dust, chemicals, toiletries, liquid cosmetics, great temperature fluctuation and closeness to sources of heat (open fires, radiators).
- Repairs may only be performed by Beurer Customer Service or by accredited retailers.
- All our personal scales comply with EU guideline 2004/108 + relevant supplements. Should you have any questions regarding operation and usage of our appliances, please contact your local retailers or Beurer Customer Service.
- Cleaning: You can clean the scale with a damp cloth and a little washing up liquid, if required.
Never immerse the scale in water or rinse it under running water.
- Storage: Do not place any objects on the scale when not in use.
- Strong electromagnetic fields (e.g. cell phones) may impair the accuracy of the scale.
- The scales can accept a maximum load of 44 lb (20 kg/704 oz). The weight measurement is displayed in 0.20 oz (5 g/0.02 lb) divisions.
- The scale is not intended to be used for commercial purposes.
- Remove transport safety catch.
- Place the scale on a level and solid surface; this is important for ensuring that the scale weighs correctly.

2. Operation

2.1 Battery

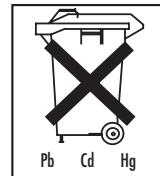
The scale requires 2 x 1.5 V alkaline mikro AA batteries. Before using the baby scales, these must be inserted in the battery compartment beneath the scales, ensuring the right polarity. When “Lo” appears in the display, the batteries need to be replaced.

Spent batteries and rechargeable batteries do not constitute normal household waste!

They are considered to be toxic waste and, as such, should be disposed of in special containers, toxic waste collection points or brought to electrical goods dealers.

Note: Batteries containing pollutant substances are marked as follows:

Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.

2.2 Weighing

The weight is displayed in steps of 5 g from a minimum weight of 50 g.

Press the button “” to switch on the scale.

As soon as “0.000 kg” appears on the display, the scale is ready to take measurements.

Now place the baby on the weighing area. Do this very carefully; do not let the baby fall and take particular care with the baby's head.

This baby scale has a “HOLD” function (AUTO HOLD and MECHANICAL HOLD). The AUTO HOLD function establishes the baby's weight on the display as soon as he/she is still enough to achieve an exact measurement value. If the baby is moving around vigorously, his/her weight cannot be recorded. Therefore, try to calm the wriggling baby to achieve a rapid measurement, but do not touch the baby, as this will affect the weight measurement.

As soon as the baby's weight is established on the display, the word “HOLD” stops flashing and the weight is displayed for a further 20 seconds. The scale then switches off automatically.

If the baby is wriggling about too vigorously, you can select the MECHANICAL HOLD function by pressing the “HOLD” button. Within 3 seconds, the word “HOLD” stops flashing and the weight is displayed for a further 20 seconds.

If you do not want to place the baby directly on the weighing surface, but prefer to use a towel, for example, you have to place the towel on the scales BEFORE switching them on. If the towel (and any other items which you want to deduct again during weighing) weighs over 50 g, you can also use the TARA function. Once you have pressed the TARA button, ‘Tare’ is shown in the display. To reset the scale to 0.000 g, press the TARA (tare) button after “HOLD” has stopped flashing.

On delivery the scale is set at “kg” units. The button “kg/lb/oz” allows you to switch at any time between kilogramme units and pounds (lb) and ounces (oz).

2.3 Disposal

Please dispose of the device in accordance with the directive – WEEE (Waste Electrical and Electronic Equipment). If you have any queries, please refer to the local authorities responsible for waste disposal.

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