

# ANIOXYDE 1000

## INDICATIONS

50 bandelettes de contrôle du taux d'acide peracétique dans la solution reconstituée d'ANIOXYDE 1000.

## MODE D'EMPLOI

- Tremper la bandelette pendant une seconde, la sortir en la secouant pour éliminer l'excès de liquide.
- Lecture après 10 secondes.
- Ne jamais interpréter un résultat obtenu plus de 15 secondes après trempage.
- Pour la fréquence d'utilisation, se reporter au mode d'emploi de l'étiquette de l'ANIOXYDE 1000.

## CONDITIONS DE STOCKAGE

- Refermer le tube après prélèvement d'une bandelette.
- Conserver à l'abri de l'humidité.
- Respecter la date limite d'utilisation reprise sur l'étiquette.
- Se conserve 90 jours après ouverture.
- Stocker entre +4°C et +30°C.

## PRÉCAUTIONS D'EMPLOI

Respectez les précautions d'emploi.  
Dispositif médical de classe I.  
Avant toute utilisation, lisez l'étiquette et les informations concernant le produit.

## INDICATIONS

50 test strips for the level of peracetic acid in the ANIOXYDE 1000 reconstituted solution.

## INSTRUCTION FOR USE

- Dip the strip for one second, remove it while shaking it to remove excess liquid.
- Reading after 10 seconds.
- Never interpret a result more than 15 seconds after dipping.
- For the frequency of use, refer to the instructions on the ANIOXYDE 1000 label.

## STORAGE CONDITIONS

- Close the container after removing each testing strip.
- Keep away from moisture.
- Respect the expiry date shown on the label.
- Store up to 90 days after opening.
- Store between +4°C and +30°C.

## PRECAUTIONS FOR USE

Respect the precautions for use .  
Class I medical device .  
Before every use, read the label and information concerning the product.



**NOIR UNIFORME SUR TOUTE LA PLAGE RÉACTIONNELLE: CONFORME**  
**UNIFORM BLACK ACROSS THE ENTIRE REACTION RANGE: COMPLIANT**



**COLORATION NON UNIFORME, PRÉSENCE DE POINTS BLANCS,**  
**ÉVANESCENCE RAPIDE DE LA COULEUR: NON CONFORME**  
**NON-UNIFORM COLOURING, EVIDENCE OF WHITE SPOTS,**  
**COLOUR SOON FADES: NON COMPLIANT**