# BOITACOUCOU / COUCOUBOX

CAISSON DE FORMATION À LA TECHNIQUE DE FRICTION / RUBBING TECHNIQUE TRAINING BOX

## NOTICE DE MONTAGE / ASSEMBLY INSTRUCTIONS

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Unpack the cardboard.

Content: 1 training box, 2 lamps, 6 batteries, 4 feet.

Assembly instructions are available below the box.

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*For lower box: Fold face 4 then put strips into the notches (Face numbers are printed on each of them).*

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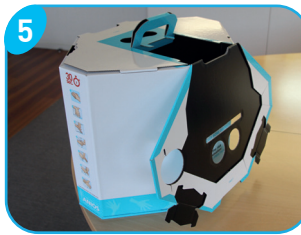
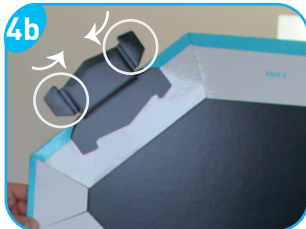
*a) Put face 3 above face 4.*

*b) Fold the ends of strips then put them into the notches.*

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
*For upper box: Repeat the same operation - fold face 2 then put strips into the notches. Fold the handle.*

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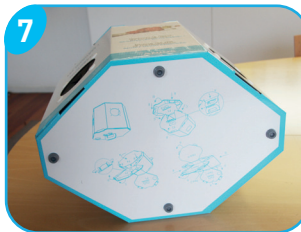*Put face 1 above face 2, put the handle through the hole in the center. Fold the ends of strips then put them into the notches.*

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*Turn the box, and stick the plastic feet below using the guides.*

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*a) Insert batteries into lamps. Place lamps heads down while using the box.*

*b) After use, place lamps heads up in order not to fall during transport. Put ANIOSGEL 85 NPC FLUORESCENT bottle in the space above the box. COUCOUBOX is ready to use.*

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*Analyse result with trainer in the double-viewing slot. LED lamps reveal areas where gel has been applied.*

*Do not hesitate to adjust hands position inside the box in order to find the better angle for viewing (up and down, right and left, nearest or farthest to you).*

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*Take a sufficient amount of ANIOSGEL 85 NPC FLUORESCENT to keep your hands damp for 30 seconds.*

*Rub using the hygienic rubbing technique (protocol is printed on each side of the box).*

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*Store in a clean and dry place. Do not spray any product directly on the cardboard. Use a slightly wet mop.*

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- *In order to ensure an optimal viewing, we recommend to replace batteries every 2 hours by new, high quality batteries.*
- *Turn off lamps after each use.*