Contour Bed Leg Rest/Raiser

VG885

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Contour Bed Leg Rest/Raiser with Memory Foam from Aidapt. The Contour Bed Leg Rest/Raiser is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

- Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.
- Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product as it may compromise your safety, but contact your supplier for support.

INTENDED USE

- The Contour Bed Leg Rest/Raiser with Memory Foam has been designed to offer improved comfort to those who need or prefer to have their legs raised and supported in bed.
- The Contour Bed Leg Rest/Raiser simply fits onto a bed or flat surface and the legs and feet can be lifted onto the Contour Bed Leg Rest/Raiser.
- No fixings are required; the Contour Bed Leg Rest/Raiser can simply be removed and stored when not in use.
- Avoid sharp objects such as zips and keys coming into contact with the cushion as this may damage the cover or foam filling.

CLEANING

Clean your Contour Bed Leg Rest/Raiser with Memory Foam using a mild detergent with a slightly damp, soft cloth only. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Do not soak the foam inner. The cover can be removed and washed at a low heat with similar colours.

After cleaning, leave the cushion and cover to dry in a warm, dry place, but not in direct contact with a heat source.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk

Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)