



Hands Free Hair Dryer Stand



VM800

Usage and Maintenance Instructions

INTRODUCTION

Thank you for purchasing the Hair Dryer Stand from Aidapt.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.


INTENDED USE

The Hair Dryer Stand is designed for users who have limited mobility.


ASSEMBLY INSTRUCTIONS

PARTS


- 1x Base **(a)**
- 1x Flexible Goose Neck Arm with Hook and Loop Fastener Strap **(b)**
- 1x Wide Pole **(c-1)** -Narrow Pole **(c-2)**
- 4x Screws

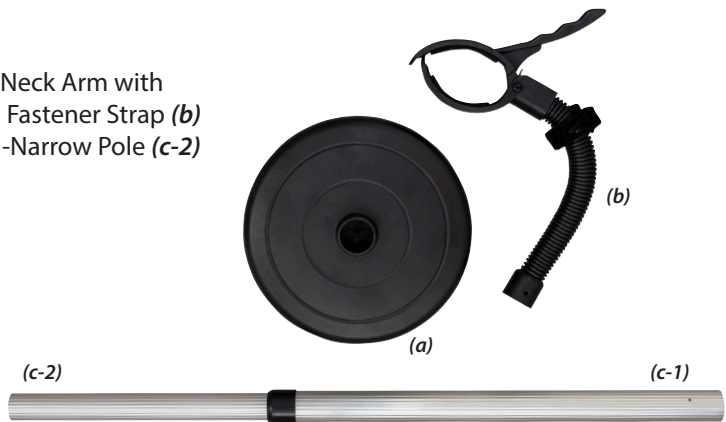

(fig. 1)

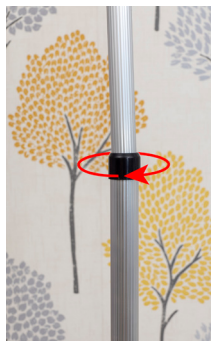

(fig. 2)

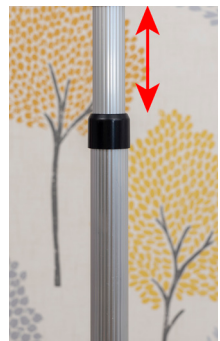

(fig. 3)

1. Place stem into base. Ensure black collar is facing up. **(fig. 1)**
2. Align screw hole in the base collar. Secure using screw provided. Ensure this is tightened before use. **(fig. 1)**
3. To adjust height, turn inner tube clockwise and set to desired height. Turn inner tube anti-clockwise to lock height in place. **(figs. 2 & 3)**



(fig. 4)



(fig. 5)

4. Place gooseneck over inner tube. Align screw holes and use screw provided to secure. Ensure this is tightened before use. **(fig. 4)**
5. Use clamp to hold hair dryer in place. **(fig. 5)**
6. Wrap the hook and loop fastener strap around the hair dryer handle to secure hair dryer in place. **(fig. 5)**
7. Bend the gooseneck arm to adjust to desired angle. DO NOT bend goose neck over 90 degrees.

CLEANING

The Hair Dryer Stand should be cleaned using a non-abrasive cleaner or mild detergent with a soft cloth only: these cleaning instructions do not apply to the hair dryer itself. Abrasive cleaners or abrasive cleaning pads may seriously damage the product beyond repair and should not be used.

TECHNICAL INFORMATION

Width 190 mm
Depth 190 mm
Height 660 mm
Net Weight 0.96 kgs

MAINTENANCE

Please check the hair dryer stand periodically for any signs of damage and if in doubt contact your supplier.

IMPORTANT INFORMATION

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product. A copy of the instructions leaflet can be downloaded from our website.

Always turn electrical items off at the socket when not in use.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX

Telephone: +44 (0) 1744 745 020 Fax: +44 (0) 1744 745 001

Web: www.aidapt.com

Email: sales@aidapt.co.uk

