
Foot Cleaner with Pumice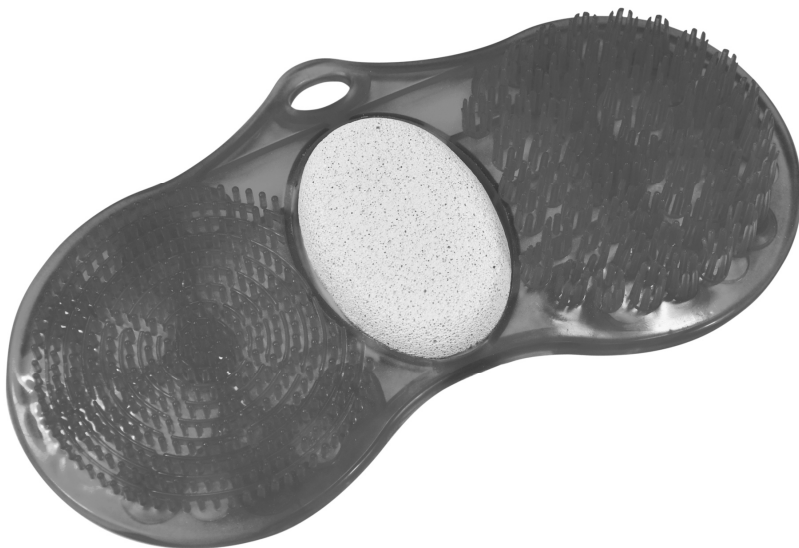

VM972

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Foot Cleaner with Pumice from Aidapt. The Foot Cleaner is manufactured from the finest materials available. When used correctly it is designed to give many years of reliable, trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the product.

Please examine your product for any visible damage before using/installing. If you see any damage or suspect a fault, please do not use, but contact your supplier.

INTENDED USE

The Foot Cleaner with Pumice is ideal for people who have limited mobility, enabling users to easily clean their feet without bending. The suction caps on its underside help to keep it safely secured to the bath during use. It can be used with any type of liquid soap and the centre pumice gently exfoliates and removes hard skin on your feet.

INSTRUCTIONS FOR USE

We recommend that you use the Foot Cleaner in the bath or shower, after the feet have been well soaked, for best results.

NB. Ensure that you are balanced by holding onto a secure, fixed rail or surface, if necessary, during use.

- Ensure the surface you intend to use the Foot Cleaner on is free from oily residue, which can prevent the suction cup from gripping.
- Position the Foot Cleaner and push down a little until the suction cups have gripped the surface.
- Raise one foot and move it over the brush or pumice as required. Swap and repeat with the other foot.

CLEANING

Clean your Foot Cleaner using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

NB. If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)