**VM701**

**Aidapt Lotion Applicator**  
**Applicateur de Lotion Aidapt**

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## INTRODUCTION

Thank you for deciding to purchase the Lotion Applicator from Aidapt.

**NB. A RISK ASSESSMENT SHOULD BE UNDERTAKEN AS TO THE SUITABILITY OF THE PRODUCT FOR THE PARTICULAR USAGE.**

## BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage if you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## INTENDED USE

This Lotion Applicator from Aidapt is designed to aid and assist in the application of creams and lotions in hard to reach areas such as shoulders and backs.

## INSTRUCTIONS FOR USE

Unscrew the round applicator head containing the roller balls. Squeeze your cream/lotion into the round tray. Do not overfill. Screw applicator lid back on ensuring it is closed tightly. Grip the Lotion Applicator with the ergonomically designed handle in either hand. When you are ready massage the round roller balls onto desired area of skin. Lotions/creams will be dispensed from the applicator head evenly.

## CLEANING

Unscrew applicator head and clean using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used.

Always ensure that you wipe dry the equipment after cleaning.

## REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

## CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

## IMPORTANT INFORMATION

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

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**Aidapt Bathrooms Ltd,**
**Altai France (Altai Europe Ltd)**