Benny / Benny XL

BADEBRETT - SHOWER BENCH - PLANCHES DE BAIN - *BADPLANK*

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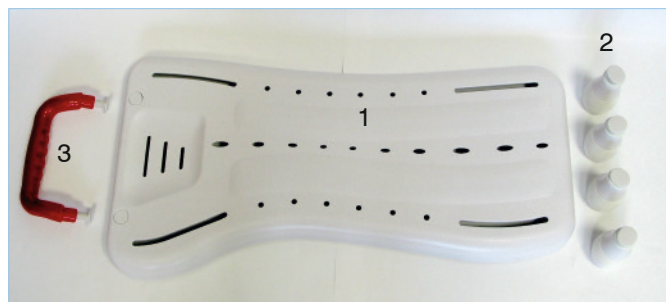
Shower benches are hung into or laid onto the frame of the bath tub. Less power is necessary because the person's lifting or lowering distance now is reduced. They reduce the required power for the person's lifting or lowering distance. Suitable if functions of upper and/or lower extremities are reduced significantly. With use of this aid bathing autonomous/independent can be done. Pre-condition/requirement: sufficient operational capability concerning the torso and the use of extremities for changing the position on the shower bench.

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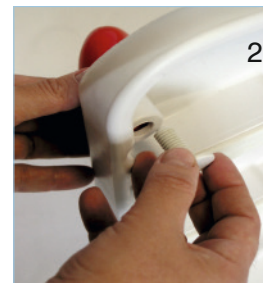
1 x Shower bench (1), 4 x Stop pins incl. fixing screws and rubber rings (2), 1 x Handgrip incl. 2 fixing screw (3). For assembling no special tool is required.

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
If necessary, remove the two plastic screws from the handgrip. Insert the fixing screws for the hand grip in the holes from the underside. The white plastic cover comes then out. Insert hand grip from above into the two openings – showing inside (see picture 1) - on the shower bench. Now tighten the plastic screws in order to fix the hand grip. Use therefore e.g. a coin (see picture 2).

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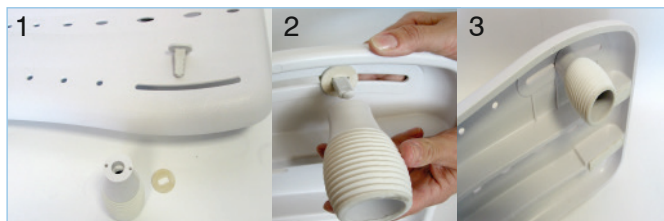If necessary, remove plastic screws and rubber ring from the stop pin. Insert in each slot of the shower bench a plastic screw from above and attach a rubber ring on it from the underside (picture 1+2), then screw the stop pin on the underside (picture 3). Do not tighten the screws completely, then the stop pins can be later adjusted within the slot on the width of the bath tub.

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Put shower bench cross on the bath tub frame. Shower bench must bear on properly. Hand grip must show to the wall. Push/Adjust stop pin on each side towards the bath tub wall so that the shower bench is fixed properly. Now fix the screws completely.

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