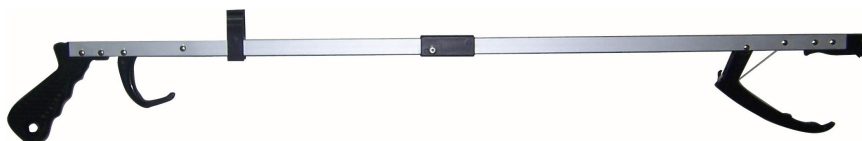
Handy Reachers

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase this Handy Reacher from Aidapt. Our Handy Reachers are manufactured from the finest materials and components available. When used correctly they are designed to give many years of reliable trouble-free service.

VM900	Handy Reacher (32 ½"/825 mm)
VM901	Handy Reacher (26"/660 mm)
VM901B	Folding Handy Reacher (32 ½"/825 mm)
VM901D	Folding Handy Reacher (26"/660 mm)
VM901F	Deluxe Handy Reacher (24"/610 mm)
VM901H	Deluxe Handy Reacher (30"/762 mm)
VM901K	Deluxe Handy Reacher (32"/813 mm)
VM901M	Deluxe Handy Reacher (35"/889 mm)

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INTENDED USE

The reacher is designed to assist in reaching and lifting items from the floor, table or shelf. It can also be used in some situations as a dressing aid; although we would stress caution is necessary when using the reacher with tights or stockings due to the fine nature of the material of these garments.

INSTRUCTIONS FOR USE

Using the reacher is simple, place the grip over the object you wish to lift and squeeze the trigger. Ensure the moving finger of the grip and the main frame of the reacher are properly secured round the object and lift.

Place the object you have lifted safely onto a flat and level surface and release the trigger.

Always assess the suitability of an item before picking it up; do not attempt to carry heavy or fragile items with your reacher.

Do not under any circumstances attempt to use the reacher to lift or move hot objects, this could lead to serious injury.

CLEANING

Clean your Handy Reacher using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may seriously damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.

If in any doubt, please do not issue or use, but immediately contact your supplier for service support.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

- Specifically, check the tension of the cable string, which attaches to the trigger and gripper (finger) section of the reacher.
- If for any reason the cable string has become slack, carefully undo the string, readjust the tension and re-tie the string.
- Squeeze the trigger to ensure that the gripping arm fits tightly, safely and square to the main frame. If not, repeat the steps above.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

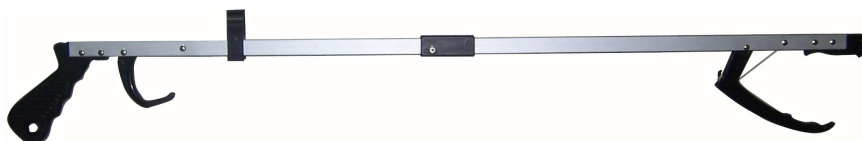
Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)