D	Hörverstärker
	Gebrauchsanweisung 2-15
GB	Sound amplifier
	Instructions for use 16-27
F	Amplificateur auditif
	Mode d'emploi 28-40
E	Amplificador
	Instrucciones de uso 41-52

DEUTSCH

Inhalt

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Contents

Included in delivery	16	5. Unit description	22
1. Intended use.....	17	6. Initial use and operation	22
2. Signs and symbols	18	7. Maintenance and cleaning	24
3. Getting to know your instrument	18	8. Disposal.....	25
4. Warnings and safety notes.....	19	9. Troubleshooting.....	26
		10. Technical specifications	27

Included in delivery

- 1 sound amplifier
- 3 earplugs in different sizes
- 2 LR754 button cell batteries
- With box for storage
- These instructions for use

Dear Customer

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, blood glucose, body temperature, pulse, gentle therapy, beauty, massage and air.

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

With kind regards
Your Beurer team

1. Intended use

The sound amplifier is only to be used to help people to hear better. The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

The sound amplifier is an amplifying device and is not a therapeutic medical product.

2. Signs and symbols

The following symbols appear in these instructions:

WARNING

Warning instruction indicating a risk of injury or damage to health.

IMPORTANT

Safety note indicating possible damage to the unit/ accessory.

Note

Note on important information.

3. Getting to know your instrument

The sound amplifier is a device to help people hear better. It does this by amplifying sounds and then transmitting these to the ear. The sound amplifier can neither prevent nor alleviate organically caused hearing loss.

The sound amplifier offers you a variety of applications. Regardless of whether you want to watch a film, chat with friends or just feel like listening to the sounds of nature in the woods, the sound amplifier helps you to hear sounds louder and more clearly.

4. Warnings and safety notes

WARNING

- Do NOT use the sound amplifier if you are suffering from an inflammation of the middle ear or ear canal.
- Do NOT use the sound amplifier at maximum volume over a longer period of time. This could damage your hearing.
- If you feel unwell or experience pain when wearing the sound amplifier, stop using it immediately and contact your doctor.
- If your ear becomes inflamed or your skin irritated or you experience an increased accumulation of ear wax when wearing the sound amplifier, stop using it immediately and contact your doctor.
- This device is not intended for use by people (including children) with restricted physical, sensory or mental skills or a lack of experience and/or a lack of knowledge, unless they are supervised by a person who has responsibility for their safety or they receive instructions from this person on how to use the device. Children should be supervised around the device to ensure they do not play with it.

- Do NOT use the sound amplifier in potentially explosive and hazardous environments.
- Keep the sound amplifier, the accessories and the batteries away from children, as there is a risk of choking.
- Keep batteries out of the reach of children. Children can put batteries in their mouth and swallow them. This may severely damage their health. In such cases, seek medical assistance immediately!
- Normal batteries must not be charged, heated or thrown into an open fire (risk of explosion!)
- Leaking or damaged batteries may burn the skin therefore use suitable protective gloves if this is the case.
- If you have any doubts, do not use a damaged sound amplifier; contact your retailer or the specified Customer Services address.
- Switch the device off immediately if it is faulty or not working properly.
- Do not attempt to open and/or repair the sound amplifier yourself. Repairs may only be carried out by Customer Services or authorised retailers. Failure to comply will result in voiding of the warranty.


IMPORTANT

- Do NOT wear the sound amplifier while bathing, showering or swimming.
- Do NOT wear the sound amplifier while applying cosmetics (e.g. make-up, hairspray, perfume or sun cream).
- Do NOT wear the sound amplifier while undergoing shortwave therapy or medical treatments that are subject to strong radio frequency or magnetic fields.
- Do NOT wear the sound amplifier in the vicinity of strong electromagnetic fields or X-rays.
- Protect the device from impacts, humidity, dirt, marked temperature fluctuations and direct sunlight.
- Never dry your sound amplifier in the microwave.
- Service your sound amplifier at regular intervals. Make sure there are no foreign objects in between the earplug and headphones, as this could impair the sound.
- Batteries must not be taken apart or short circuited.
- Leaking batteries may damage the device. If you do not intend to use the device for longer periods, remove the batteries from the battery compartment.

5. Unit description


- 1 Microphone
- 2 Volume control
- 3 On/Off switch
("I" = On, "O" = Off)
- 4 Battery compartment
- 5 Headphones
- 6 Earplug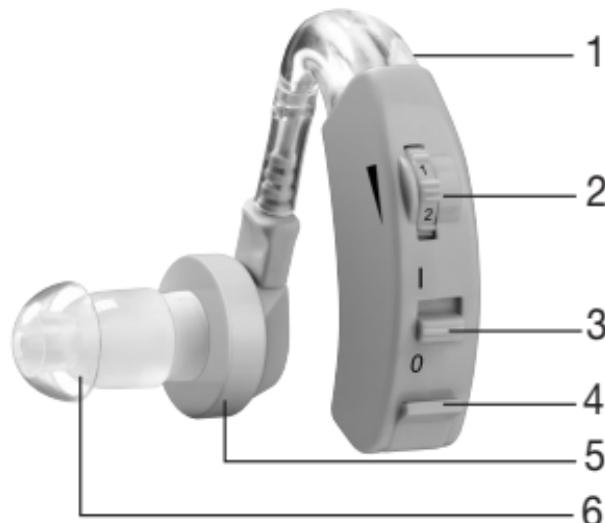


6. Initial use and operation

Remove the sound amplifier from the packaging and check that the device and all accessories are included and not damaged.


1. Open the battery compartment on the sound amplifier as shown.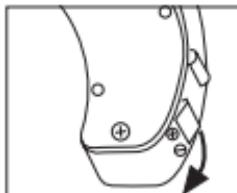

2. Insert the button cell batteries (type LR754) included in the delivery into the battery compartment as shown. Ensure when inserting the batteries that the correct polarity (+/-) is observed.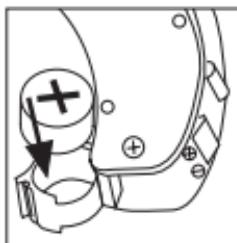

3. Close the battery compartment again. Now connect your desired earplug to the headphones.

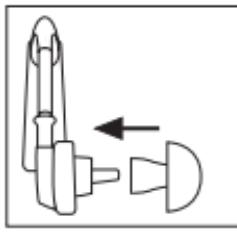i Make sure the volume control is initially set to the lowest setting before operating the sound amplifier.

4. Bring the sound amplifier up to your ear as shown. Slowly insert the earplug into your ear canal applying no force. Make sure not to insert the earplug too deeply into the ear canal. If the earplug is not properly in place or feels uncomfortable, try a different earplug.
5. Move the On/Off switch to “I” to switch on the sound amplifier. Use the volume control to adjust the volume to the desired level so that you can hear sounds clearly and distinctly.

i If you hear a slight whistling sound when wearing the sound amplifier, check that your earplug is fitted properly in your ear. If it is not snug enough, use a different earplug.

7. Maintenance and cleaning

IMPORTANT

- Remove the batteries from the device before cleaning.
- Clean the sound amplifier and the earplugs with a clean, dry cloth after each use. If they are very dirty, you can also moisten the cloth with a mild

soapy solution. Never use abrasive, corrosive or solvent-based cleaning products.

- Allow the sound amplifier to dry fully after cleaning. Store the sound amplifier in a safe, dry and clean place.
- Under no circumstances should you hold the sound amplifier under water, as this can cause liquid to enter and damage the sound amplifier.
- If you do not intend to use the sound amplifier for longer periods, remove the battery from the device to prevent possible leaking.

8. Disposal


Please dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any queries, please contact the appropriate local authorities.

The empty, completely flat batteries should be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

Note: The codes below are printed on batteries containing harmful substances: Pb = Battery contains lead, Cd = Battery contains cadmium, Hg = Battery contains mercury.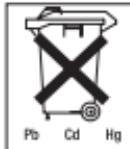

9. Troubleshooting

Problem	Possible causes
No sounds can be heard from the sound amplifier.	<ul style="list-style-type: none">- Sound amplifier is not switched on.- Batteries are not inserted correctly.- Battery is empty.- Microphone is dirty.- Sound amplifier is damaged.
The sound amplifier is very quiet.	<ul style="list-style-type: none">- Volume setting on sound amplifier is too low.- Battery is almost empty.- Earplug is dirty.- Microphone is dirty.

10. Technical specifications

Maximum audio output: 128dB	Voltage: 1.5 V
Sound amplification: max. 40dB	Overall harmonic distortion: ≤5%
Frequency range: 200 – 5000Hz	Power supply: ≤4mA
Input noise: ≤30dB	

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