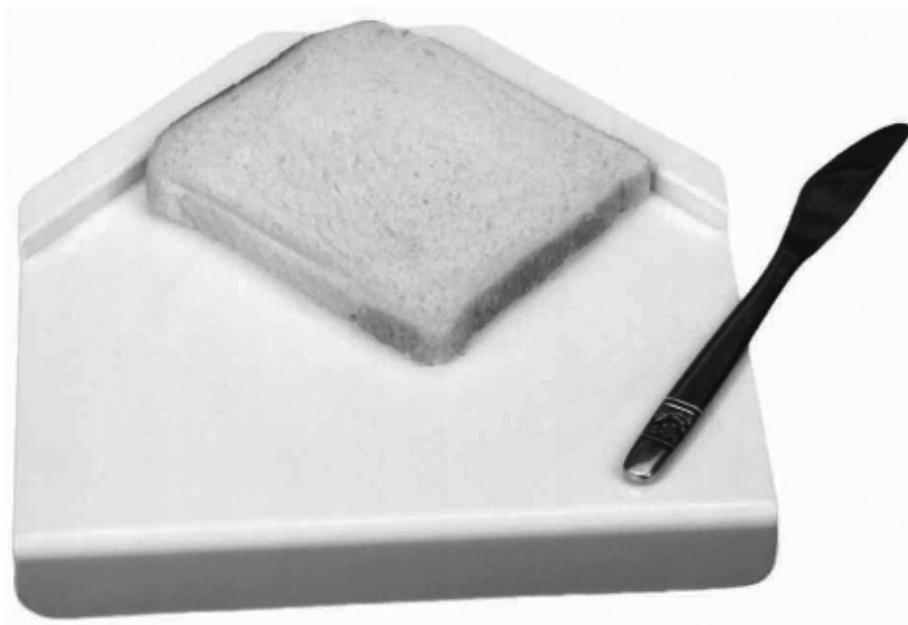
## **Preparation Board**

**VM982**

## **Usage and Maintenance Instructions**

## INTRODUCTION

Thank you for deciding to purchase the Preparation Board from Aidapt. The Preparation Board is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

## BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

## INTENDED USE

Food preparation board for those with limited hand use.

## INSTRUCTIONS FOR USE

The Preparation Board is simple to use:

- Position the Preparation Board onto the work surface with the straight lip over the edge of the work surface to hold the Preparation Board in place.
- The Preparation Board features non-slip feet, which will help to keep it in place.
- The rear upstand of the Preparation Board will hold bread and other food steady for buttering and cutting, etc.

## CLEANING

Clean your Preparation Board using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

The Preparation Board can also be cleaned on the top tray of a domestic dishwasher on a standard/eco or cool program.

## REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

**If in any doubt, please do not issue or use, but immediately contact your supplier for service support.**

## CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

## IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

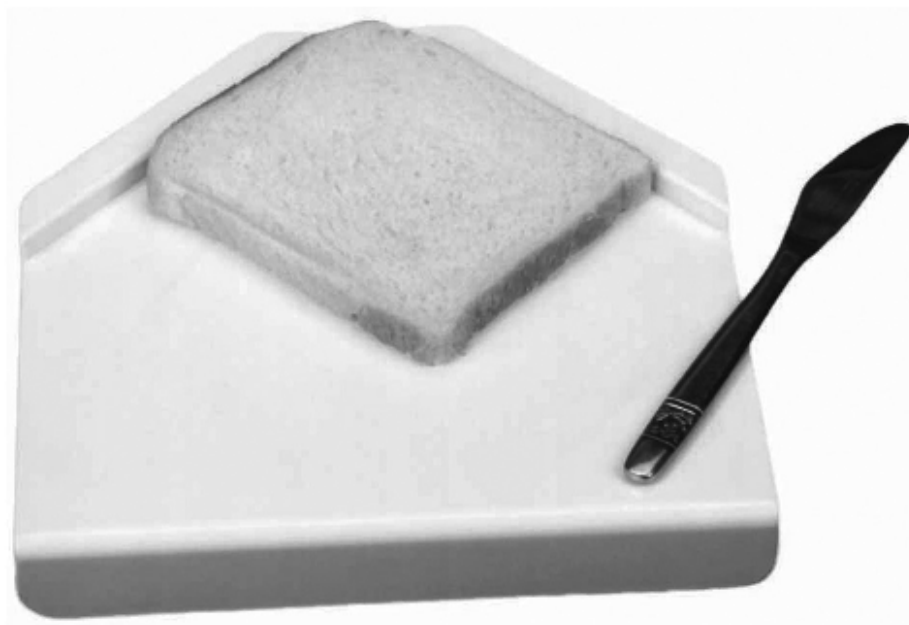
Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

**Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX**  
**Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: [www.aidapt.co.uk](http://www.aidapt.co.uk)**  
**Email: [accounts@aidapt.co.uk](mailto:accounts@aidapt.co.uk) • [adaptations@aidapt.co.uk](mailto:adaptations@aidapt.co.uk) • [sales@aidapt.co.uk](mailto:sales@aidapt.co.uk)**

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**Altai Germany (Altai Europe Ltd.)**