**VM909A
Kettle Tipper**

This file is available to view and download as a PDF at www.aidapt.co.uk. Sight impaired customers can use a free PDF Reader (such as adobe.com/reader) to zoom in and increase the text size for improved readability.

Fixing and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase the Kettle Tipper from Aidapt. The Kettle Tipper is manufactured from the finest materials and components available. When used correctly it is designed to give many years of reliable trouble-free service.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the product.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

INSTRUCTIONS FOR USE

Using the Kettle Tipper is simple:

- Position the upright kettle against the round wire frame and use the fixing strap to secure the kettle below the handle.
- The wire frame design means that your cup can be placed as near as required to the kettle.
- Tip the kettle towards the cup using the handle, but allowing the Kettle Tipper to take the weight of the kettle.
- Gently return the kettle to the upright position, allowing the Kettle Tipper to take the weight of the kettle.
- The kettle will return to its upright position when released due to the Kettle Tipper's raised pivot point, which prevents spillage.

CLEANING

Clean your Kettle Tipper using a non-abrasive cleaner or mild detergent with a soft cloth. Abrasive cleaners e.g. AJAX and/ or abrasive cleaning pads may damage the product beyond repair and should not be used. Always ensure that you wipe dry the equipment after cleaning.

REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.

IF IN ANY DOUBT, PLEASE DO NOT ISSUE OR USE, BUT IMMEDIATELY CONTACT YOUR SUPPLIER FOR SERVICE SUPPORT.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.com
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai France (Altai Europe Ltd)