ROAD

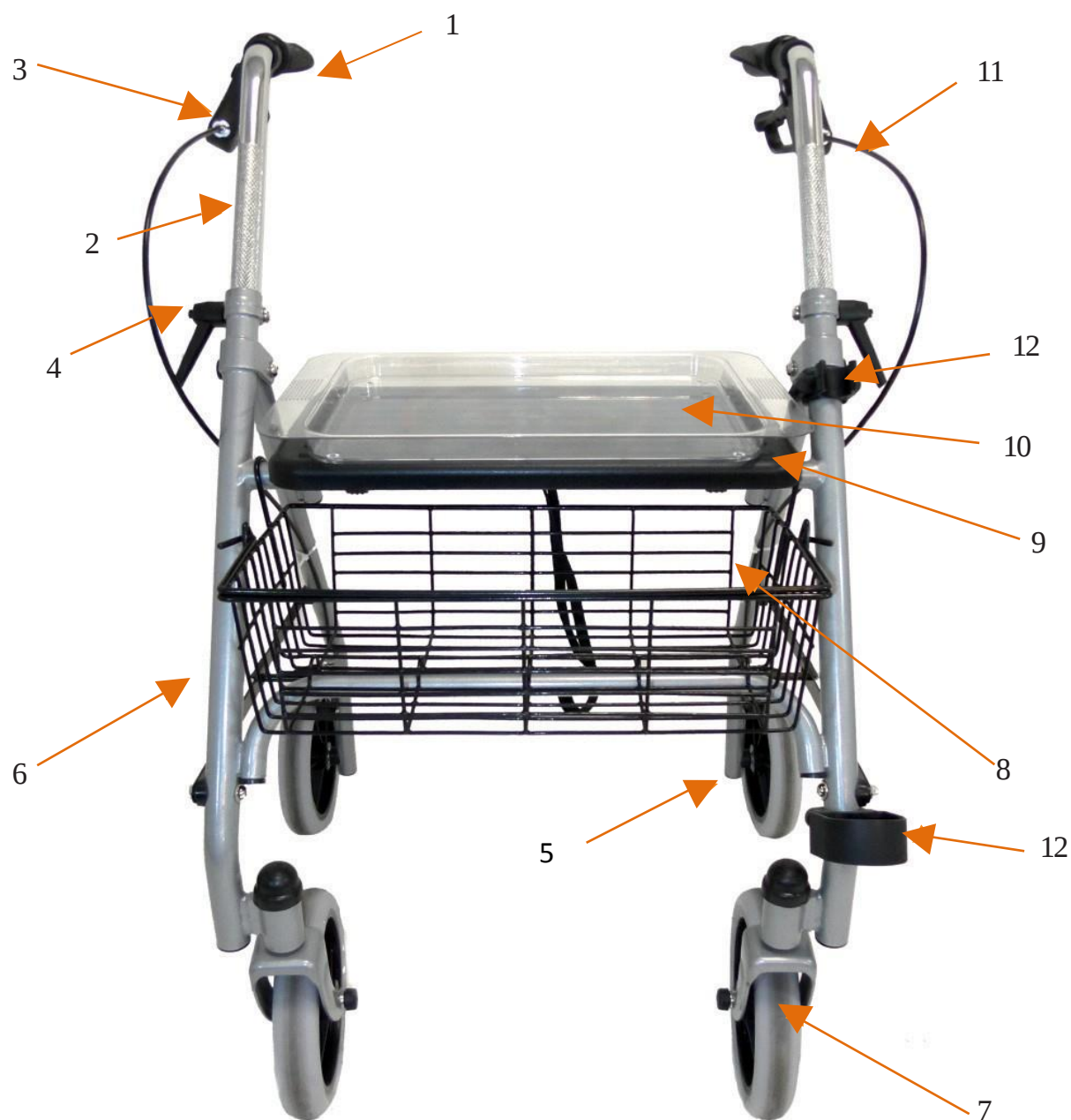
ROLLATOR

REF SA1311005

Manuel d'utilisation – Operating instructions
Bedienungsanleitung – Gebruikshandleiding

Composants du Rollator - Parts of the Rollator - Teile des Rollators - Onderdelen van de Rollator

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Height adjustment handgrips

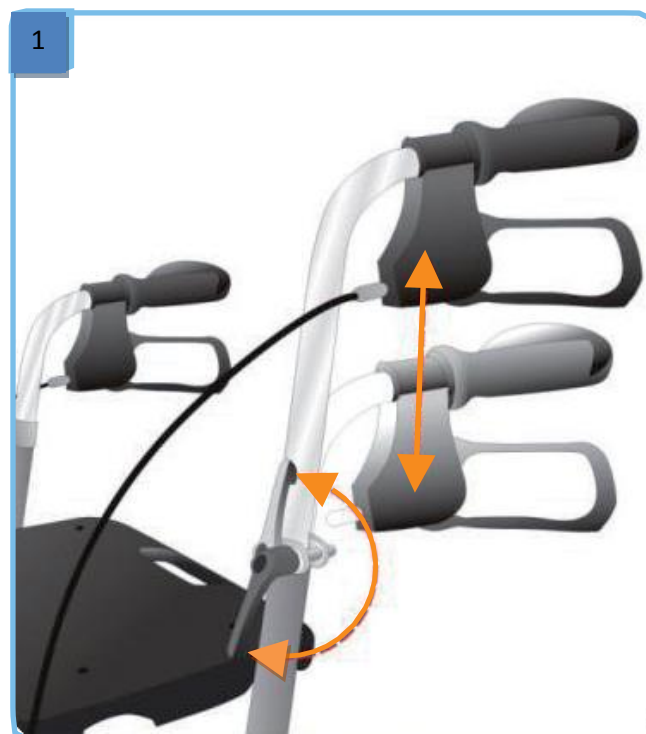
Insert the tube of the adjustable handgrip into the frame at least to the "Maximum" marking.

Fix it in the desired height with the fixing lever by turning it clockwise (see picture1).

Adjust both handgrips on the same height / level. Handgrips must be adjusted parallel to the walking direction!

Fixing lever itself can be adjusted in the desired position by simultaneously pulling and twisting.

For readjustment of the height of the handgrips unlock the fixing lever by turning it counterclockwise. Then fix the handgrip how described above.

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Position the walking device on its wheels. Push the front frame forward by pressing on the handgrips down and pulling backwards until the walking device stands on four wheels.
Then move the seat down until it touches the crossbar.
(see picture 2)

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Hook in the basket in the hooks at the frame under the seat plate and place the tray with its small feet in the holes in the seat plate (see picture 3).

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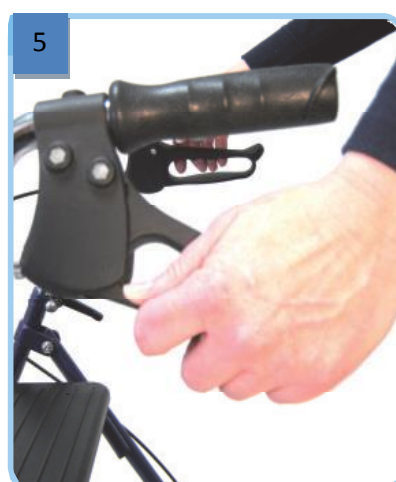
For breaking the walking device pull both brakes upwards to the handgrip (see picture 4). Always brake slowly and steady.

For parking the walking device press down the brakes until they click in the parking position (see picture 5).

For unlocking the brakes pull them upwards towards the handgrips until they are unlocked (both brakes are in a middle position).

The effect of the brakes can be affected by abrasion of the wheels.

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For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 7) or with the suitable tool (not included) on the brake wire (picture 6). The gap between brake plate and tire should be between 3-4 mm (see picture 8).

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Position the walking device on an even and solid surface.

Then lock the brakes (parking position). Walking device must stand and must not slip. Sit straight up and do not lean backwards (see picture 9).

Seating position only as shown allowed (against the direction)!

Pliage et rangement – Folding and storage

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For folding pull seat upwards (see pictures 10, 11 and 12). Walking device can now be transported by trailing or is now suitable for storage (picture 13).

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Use only as a walking device.

Use only on even and solid surfaces.

Walking device must not be used for standing on or as a wheelchair.

It must not be used for transport of persons or animals.

Maximum load Walking device: 120 kg

Use tray only for small, light, objects. Max. load: 5.0 kg

Use basket only for transport of objects and do not overload. Max. load: 5.0 kg

Surface temperature

If the wheelchair is exposed to a heat source, such as a heater or the sunrays, either directly or through a windowpane, or is in a car boot, the temperature of the upholstery, and metallic and plastic parts may be very high and burn the user. Those elements should be cooled before using the Rollator.

Nettoyage/Entretien – Cleaning/Maintenance

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.....

Clean regularly by hand with a damp cloth! Do not use any aggressive detergents!

Screws must be checked regularly and if necessary retightened.

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.....

The Rollator is suitable for reuse.

Ensure that the new user is given all of the technical documents required to ensure safe handling.

The Rollator must be cleaned and disinfected prior to reuse and must be inspected for damage and approved by the retailer.

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.....

We issue a warranty of 12 months from date of purchase. Subject to technical alterations. Errors and omissions excepted.

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.....

Dimensions

Length over all(with basket)	610 mm
Width over all	590 mm
Width between handgrips	405 mm
Width between the rear wheels(free space for walking)	410 mm
Handgrip height from...to	790 to 920 mm
Seat height	600 mm
Dimensions seat L x W	220 x 390 mm
Dimensions tray (useful area) LxW	230 x 320 mm
Wheels diameter/width	190/35 mm

Dimensions folded

Height (without/with handgrip)	760/ 960 mm
Width	590 mm
Length	230 mm

Weight

Without basket and tray	8,8 kg
With basket and tray	9,6 kg
Weight basket	0,5 kg
Weight tray	0,3 kg

Max. loads

Max. load frame	120 kg
Max. load in basket/on tray	5,0 kg

Materials

Precision steel tube, high-quality plastic

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- 1 Product name
- 2 Article number
- 3 Maximum weight
- 4 Batch number
- 5 Attention: See operating instructions

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