3M™ Red Dot™

ECG Monitoring Electrode

(English)

Intended Use and Product Description:

The information provided in this insert applies to the 3M™ Red Dot™ Monitoring Electrodes 2200 and 9600 series with snap connectors. This information does not apply to the 3M™ Red Dot™ 2268 monitoring electrode and also does not apply to the 3M Monitoring electrodes 2228 and 2244. The 3M™ Red Dot™ monitoring electrodes are disposable with a silver/silver chloride sensing element and are designed for ECG monitoring. These electrodes have been tested for up to 3 days wear.

Warning:

This device has not been tested for use during Magnetic Resonance Imaging (MRI) procedures. The use of conductive patient-connected devices and patient lead wires/electrodes in MRI procedures may result in serious patient burns. If a physician orders this device to be used during MRI procedures, the user should not allow loops in patient lead wires or allow the lead wires to come in contact with the patient's skin.

Caution:

U.S.A. federal law restricts this device to sale by or on the order of the physician.

Precautions:

- To prevent dryout, electrodes should be used within 30 days of opening the pouch. The unused electrodes should remain stored in the opened pouch.
- Do not use an electrode if the gel is dry or if the gel column is dislodged from the electrode.
- For proper skin management and to minimize skin irritation:
 - Avoid placing an electrode on an irritated skin site.
 - Do not abrade a skin site more than one time.
 - Avoid removing electrodes frequently and/or reapplying to the same skin site.
 - Avoid placing electrodes on skin still wet from an alcohol wipe (dry thoroughly).
 - Assess electrode sites periodically.
- During surgical procedures, place ECG electrodes as far as possible from the electrosurgical site to minimize RF current flow through the electrode site. Otherwise burns could result at the ECG electrode site.

Instructions For Use:

- Select and prepare an electrode site according to your health care facility's protocol for ECG monitoring or diagnostic procedures.
- The electrode site should be clean, dry and free of skin oil before application of the electrode to allow optimal electrode adhesion to skin. Application sites with heavy or curly hair that cannot be parted should be clipped.
- For best trace quality, skin abrasion should be part of the skin preparation. Use 3M™ Red Dot™ Trace Prep or the electrode's built-in abrader as part of the skin preparation.
- Attach the lead wire to the electrode.
- Remove electrode from the liner. Avoid touching the adhesive surface.
- Place the center of the electrode over the prepared skin site.
- Remove the paper liner from the tab if applicable.
- Smooth the electrode down using a circular motion. Avoid pressing on the center of the electrode.
- To minimize skin trauma during electrode removal, pull up slowly and gently from the tab or the edge of the electrode.

Shelf Life:

For shelf life, refer to the expiration date that is printed on each package.

Thank you for choosing 3M™ Red Dot™ Monitoring electrodes. If you are not completely satisfied or if you have questions or comments, in the U.S.A., please call the 3M™ Health Care Customer Helpline at 1-800-228-3957. In Canada, call 1-800-364-3577.

Explanation of Symbols:

- Not made with natural rubber latex.
- Not made with natural rubber latex.
- Caution, see instructions for use
- Do not reuse

- Use by date
- Batch code
- Manufacturer
- Date of Manufacture

Made in Canada by /
Fabriqué au Canada par
3M Health Care / 3M Soins de santé
3M Canada
2510 Conway Ave
St. Paul, MN 55144-1000
www.3M.com

3M Deutschland GmbH
Health Care Business
Carl-Schurz-Str.1
D41453 Neuss, Germany

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