COUSSINET GEL
ORTEILS

EN GEL PAD
TOES

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Dear Customer,
Thank you very much for placing your trust in an Orliman product. Please read the instructions carefully. Keep these instructions and the packaging for future reference. If you have any questions or concerns, please contact your doctor, orthopaedic specialist or our customer service department.
ORLIMAN S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered except for the intended use as described in these instructions.
If the products are used in combination with other products, replacement parts or systems, make sure they are compatible and made by Orliman®. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. If any serious incidents related to the product occur, notify Orliman S.L.U. and the corresponding competent authority in your country.
Orliman would like to thank you for choosing this product and hopes you a speedy recovery.

REGULATIONS
[MD] This article is defined as a class I medical device. A Risk Analysis (UNE EN ISO 14971) has been carried out, minimising the existing risks. Tests have been realised in accordance with European Regulation UNE-EN ISO 22523 on Prostheses and Orthoses.

INDICATIONS
Relieves pain and provides comfort to the toes thanks to its viscoelastic polymer gel pad. Easily stays in place thanks to its toe loop.
POLYMER GEL TECHNOLOGY: Thanks to it is composition of medical mineral oil, moisturize, soften, protect, and cushion the affected area.

CONTENTS / DESCRIPTION
- Gel pad Toes (1 unit),
- Instructions for use, to be carefully read.

COMPOSITION
Polymer Gel.

RECOMMENDATIONS-WARNINGS
The use of this product is subject to the indications. Although this product is not for single use, it is recommended that only one product is used per patient – only for the purposes indicated in these instructions or by a healthcare professional – and that it is worn on completely dry skin if alcohol-based products (e.g. hydroalcoholic gel), ointments, etc. are used.
KEEP OUT OF REACH OF CHILDREN.

When disposing of the packaging and the product, please strictly observe the legal regulations of your region.

RECOMMENDATIONS FOR STORAGE AND WASHING
When not using the product, keep it in its original packaging, in a dry place and at room temperature. Stick the Velcro fasteners together (if any) and wash the product regularly by hand with warm water (max. 30°C) and a mild soap. To dry the product, use a dry cloth to absorb as much moisture as possible and leave it in a horizontal position at room temperature. Do not iron or expose to direct heat sources such as cookers, ovens, direct sunlight, etc. Do not use abrasive or corrosive substances or products based on alcohol, chlorine or solvents to wash the product. Rinse the product well, as any residual detergent could cause skin irritation and damage the product.

FITTING INSTRUCTIONS
For best therapeutic results considering different pathologies and to extend the useful life of the product, it is essential to choose the correct size for each patient or user. Excessive compression may be intolerable; adjusting the compression to be firm yet comfortable is recommended.
If the product must be adapted, any such adjustments must be done by an orthopaedic specialist or healthcare professional legally certified to do so who must make sure the end user or person responsible for fitting the product properly understands how it works and should be used.
Remove product at least 3 to 4 hours daily to allow skin to breathe.

PRECAUTIONS
Before each use, check that all product components are present as per the fitting process. Periodically check the conditions of the product. If you observe any defect or anomaly, immediately report it to the issuing establishment.
This product is made of inflammable material. Do not expose the products to situations that could set them on fire. In the event of a fire, quickly get them off your body and use the proper resources to extinguish the fire.
To avoid minor discomfort caused by sweating, we recommend using some type of cotton fabric to separate the skin from contact with the product material. For discomfort such as chafing, irritation and swelling, remove the product and see a doctor or orthopaedic specialist. The product should only be used on healthy skin. It is not recommended for use over open scars with swelling, redness or hotspots.
Products marked with the symbol contain natural rubber latex and can cause allergic reactions in people sensitive to latex.
Products marked with the symbol contain ferromagnetic components and, therefore, extreme precaution must be taken if you undergo an MRI scan or are exposed to radiation associated with diagnostic or therapeutic procedures.

The manufacturing date is included in the batch number which can be found on the packaging label as **[LOT]**, in the following way: the second and third digits represent the year of manufacture and the fourth and fifth digits represent the month.

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