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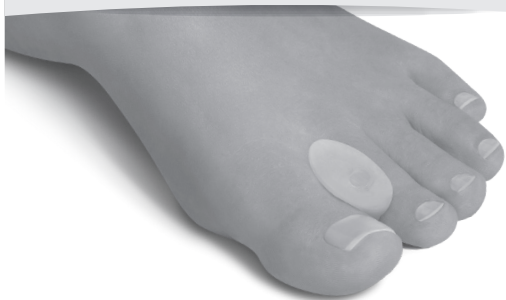
EN TOE SEPARATOR
HALLUX VALGUS

ES SEPARADOR DE DEDOS
HALLUX VALGUS

Modèles bilatéraux / Fits left or right / Modelos bilaterales.

Taille / Size / Talla	Pointure / Shoe size / Talla calzado (EU)
S	35 - 38
M	39 - 42
L	43 - 46

FR FRANÇAIS

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Dear Customer,
Thank you very much for placing your trust in an Orliman product. Please read the instructions carefully. Keep these instructions and the packaging for future reference. If you have any questions or concerns, please contact your doctor, orthopaedic specialist or our customer service department.
ORLIMAN S.L.U. guarantees all its products as long as the original configuration has not been manipulated or altered except for the intended use as described in these instructions.
If the products are used in combination with other products, replacement parts or systems, make sure they are compatible and made by Orliman®. It does not guarantee any products with altered characteristics due to improper use, defects or breakage of any kind. To use the warranty, go to the place where you bought the product as they must complete the warranty information table on this instructions sheet. If any serious incidents related to the product occur, notify Orliman S.L.U. and the corresponding competent authority in your country.
Orliman would like to thank you for choosing this product and hopes you a speedy recovery.

REGULATIONS
[MD] This article is defined as a class I medical device. A Risk Analysis (UNE EN ISO 14971) has been carried out, minimising the existing risks. Tests have been realised in accordance with European Regulation UNE-EN ISO 22523 on Prostheses and Orthoses.

INDICATIONS
Prevents overlapping toes. Helps to realign the metatarsophalangeal joint and limits deviation of the big toe. Reduces friction between the toes thanks to its moisturising and soothing polymer gel.

CONTENTS / DESCRIPTION
- Toe separator Hallux Valgus (2 units),
- Instructions for use, to be carefully read.

COMPOSITION
Polymer Gel.

The manufacturing date is included in the batch number which can be found on the packaging label as **[LOT]**, in the following way: the second and third digits represent the year of manufacture and the fourth and fifth digits represent the month.

RECOMMENDATIONS-WARNINGS
The use of this product is subject to the indications. Although this product is not for single use, it is recommended that only one product is used per patient – only for the purposes indicated in these instructions or by a healthcare professional – and that it is worn on completely dry skin if alcohol-based products (e.g. hydroalcoholic gel), ointments, etc. are used.
KEEP OUT OF REACH OF CHILDREN.

When disposing of the packaging and the product, please strictly observe the legal regulations of your region.

RECOMMENDATIONS FOR STORAGE AND WASHING
When not using the product, keep it in its original packaging, in a dry place and at room temperature. Stick the Velcro fasteners together (if any) and wash the product regularly by hand with warm water (max. 30°C) and a mild soap. To dry the product, use a dry cloth to absorb as much moisture as possible and leave it in a horizontal position at room temperature. Do not iron or expose to direct heat sources such as cookers, ovens, direct sunlight, etc. Do not use abrasive or corrosive substances or products based on alcohol, chlorine or solvents to wash the product. Rinse the product well, as any residual detergent could cause skin irritation and damage the product.

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