Garden Kneeler/Bench

VL130

Usage and Maintenance Instructions

INTRODUCTION

Thank you for deciding to purchase Garden Kneeler/Bench from Aidapt. This item is manufactured from the finest materials and components available. When installed and used correctly it is designed to give many years of reliable trouble-free service.

IMPORTANT!

NB. This equipment must be installed by a competent person.

Weight Limit: 110 kg (17 stone)

NB. DO NOT exceed the weight limit stated – doing so could put the user at risk.

BEFORE USE

Carefully remove all packaging. Avoid using any knives or other sharp instruments as this may damage the surface of the walking stick.

Check the product for any visible damage. If you see any damage or suspect a fault, do not use your product, but contact your supplier for support.

CLEANING

The Garden Kneeler/Bench should be cleaned using a non-abrasive cleaner or mild detergent with a soft cloth. Only use a mild detergent or disinfectant; never use an abrasive cleaner. Always ensure that you wipe dry all equipment and treat joints with a water dispersing agent, e.g. WD40.

CARE & MAINTENANCE

Please make a safety check of the product at regular intervals or if you have any concerns.


REISSUE

If you reissue or are about to reissue this product, please thoroughly check all components for their safety.

This includes:

- Tightness of all nuts/ bolts/castors
- Other screw in/ bolt in/ push in components.



If in any doubt, please do not issue or use, but immediately contact your supplier for service support.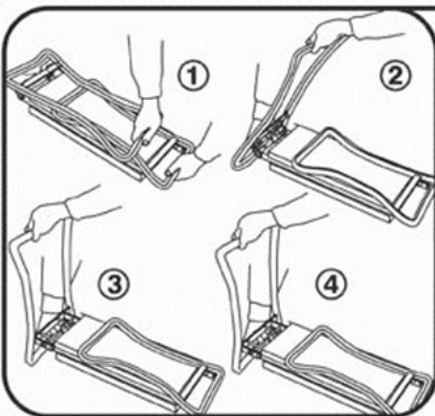

To Unfold Legs for use:

To open, place bench upside down (in kneeler position).
To unfold each leg one at a time, grasp and press release rod. Slowly pull leg outward. When fully extended, make sure release rod clicks into place.

Folding Garden Bench Instructions

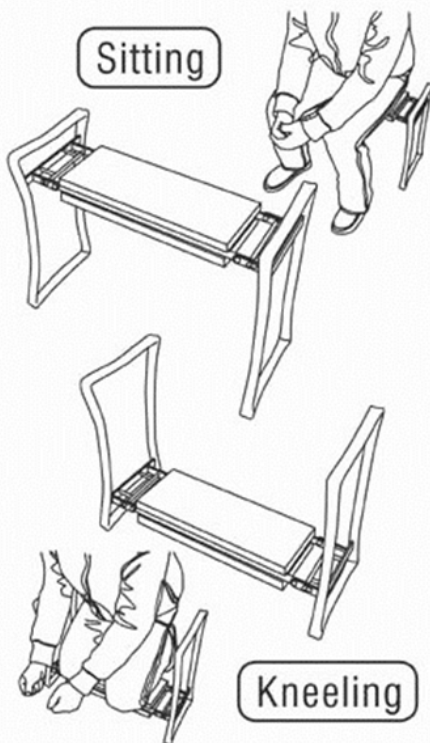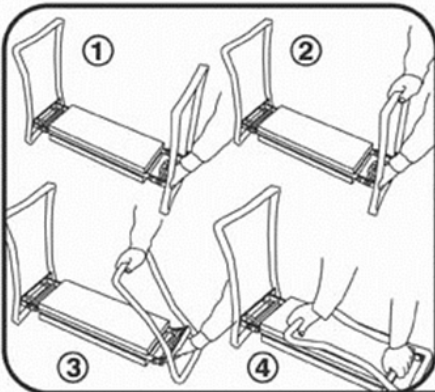To Fold Legs for Storage:

To close, place bench upside down (in kneeler position).
To Fold each leg, grasp and press release rod.
Slowly fold each leg down one at a time.

IMPORTANT INFORMATION

The information given in this instruction booklet must not be taken as forming part of or establishing any contractual or other commitment by Aidapt Bathrooms Limited, Aidapt (Wales) Ltd or its agents or its subsidiaries and no warranty or representation concerning the information is given.

Please exercise common sense and do not take any unnecessary risks when using this product; as the user you must accept liability for safety when using the product.

Please do not hesitate to contact the person who issued this product to you or the manufacturer (detailed below) if you have any queries concerning the assembly/use of your product.

Aidapt Bathrooms Ltd, Lancots Lane, Sutton Oak, St Helens, WA9 3EX
Telephone: +44 (0) 1744 745 020 • Fax: +44 (0) 1744 745 001 • Web: www.aidapt.co.uk
Email: accounts@aidapt.co.uk • adaptations@aidapt.co.uk • sales@aidapt.co.uk

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Altai Germany (Altai Europe Ltd.)