drive

DeVilbiss
HEALTHCARE

Rollator

Cristallo

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
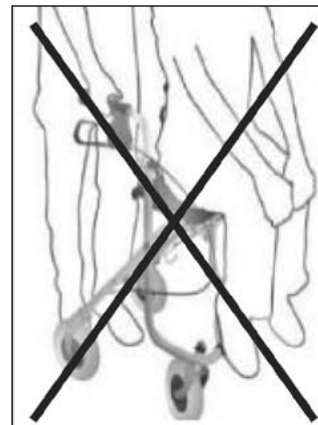
INDICATION

Walking device – for indoor and outdoor use – for compensation and improvement respectively of restricted ability of walking.

INSTALLATION

Height adjustment handgrips

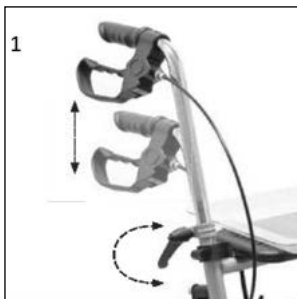Insert the tube of the adjustable handgrip into the frame at least to the “MAX” marking. Fix it in the desired height with the fixing lever by turning it clockwise. Adjust both handgrips on the same height / level. Handgrips must be adjusted parallel to the walking direction! Fixing lever itself can be adjusted in the desired position by simultaneously pulling and twisting. For readjustment of the height of the handgrips unlock the fixing lever by turning it counterclockwise. Then fix the handgrip how described above.


Unfold

Place the walking device on its wheels. By pulling one of the pipes of a back wheel you can unfold the frame (1), then press the crossbar down (2) until it easily engages. Turn the movable folding protection down (3) until this stands vertically to the crossbar.


Fitting with basket and tray

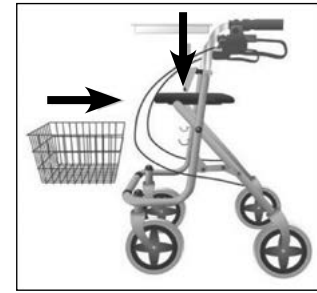Hook in the basket in the hooks at the frame under the seat plate and place the tray with its small feet in the holes in the seat plate (see picture).


BRAKING / PARKING

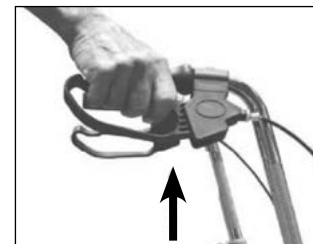For braking the walking device pull both brakes upwards to the handgrip (see picture). Always brake slowly and steady.

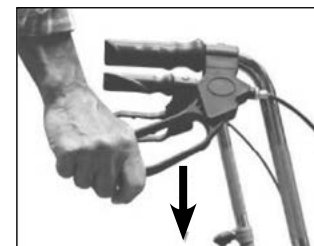
For parking the walking device press down the brakes until they click in the parking position (see picture). For unlocking the brakes pull them upwards towards the handgrips until they are unlocked (both brakes are in a middle position).

The effect of the brakes can be affected by abrasion of the wheels. Therefore test regularly the brakes.

ADJUSTMENT OF BRAKE

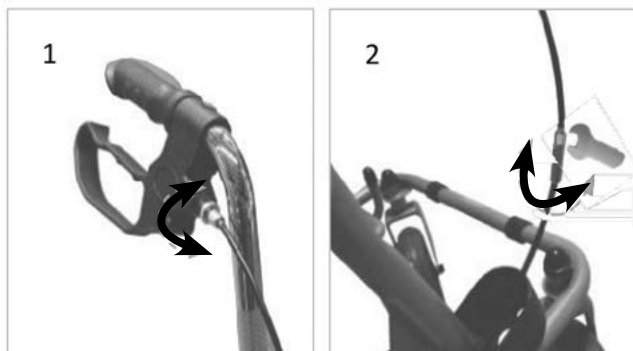For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture1) or with the suitable tool (not included) on the brake wire (picture 2).

SITTING

Position the walking device on an even and solid surface. Then lock the brakes (parking position). Walking device must stand and must not slip. Sit straight up and do not lean backwards (see picture).

 **Seating position only as shown allowed (against the direction)!**

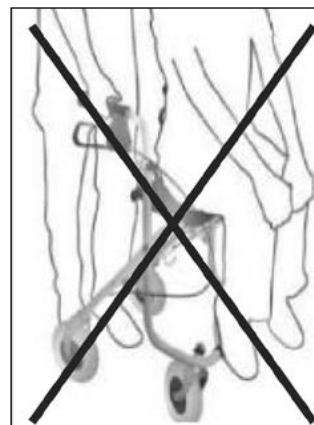
FOLDING /TRANSPORT/STORAGE

For folding open folding protection first (picture 1) then pull seat upwards (picture 2). Walking device can now be transported by trailing or is now suitable for storage (picture 3).

SAFETY INSTRUCTIONS

- Use only as a walking device !
- Use only on even and solid surfaces.
- Walking device must not be used for standing on or as a wheelchair !
- Use tray only for small, light, but not for pointed or hot objects.
- Max. load tray: 5,0 kg
- Use basket only for transport of objects and do not overload !
- Max. load basket: 5,0 kg
- Maximale Last: 130 kg

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HEBEI HEALTHPLUS MEDICAL DEVICE CO., LTD.