MADRID

Rollator

SA1312005

Teile des Rollator -
Parts of walking device -
Pièces du Rollator -
Onderdelen van de rollator -

1 Handgriff	1 Handgrip	1 Poignée	1 Handgreep
2 Schiebegriff	2 Adjustable Tube	2 Poignée réglable en hauteur	2 Hoogteverstelling
3 Bremse	3 Brake	3 Frein	3 Rem
4 Feststellschraube für Schiebegriff	4 Height Adjustment screw for handgrips	4 Molette de fixation des poignées	4 Borgschroef van hoogte- verstellingen
5 Hinterrad	5 Rear wheel	5 Roue arrière	5 Achterwiel
6 Rahmen	6 Frame	6 Châssis	6 Frame
7 Vorderrad	7 Front wheel	7 Roue avant	7 Voorwiel
8 Korb	8 Basket	8 Panier	8 Mand
9 Tablett	9 Tray	9 Tablette	9 Dienblad
10 Bowdenzug	10 Brake wire	10 Câble de frein	10 Bowdenkabel
11 Tasche	11 Bag	11 Sacoche souple	11 Tas

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Package contents

1. Walking device
2. Basket and bag
3. Tray
4. Operating instructions

Indication / Contraindication

Walking device – for indoor and outdoor use – for compensation and improvement respectively of restricted ability of walking.

The purpose of walking devices is to compensate for or at least improve restricted walking ability in users (expansion of range and/or mobility).

Persons with restricted hand/arm functions require special adjustments/versions of handgrips or hand rests/armrests.

For persons with balance disorders are walking frames not recommended, this is to be checked by an experiment in an individual case. If it is necessary a companion must be present.

Installation

Height adjustment handgrips

Insert the tube of the adjustable handgrip into the frame at least to the “first hole”. Insert the locking screw through the pipe from the inside and fix with washer and star clutch screw.

To change the height setting of the adjustable handgrip the locking screw by turning off the star clutch against the clockwise solve. Then ascertain as on top described the clutch by desired height. (see Figure1).

Unfold

To fully unfold the TRI walker before using, push the folding mechanism down away from the handles. (see Figure 2).

Fitting bag, basket and tray

Hook in the basket in the hooks at the frame under the seat plate and place the tray with its small feet in the holes in the seat plate (see picture 3 to 5).

Breaking / Parking

For breaking the walking device pull both brakes upwards to the handgrip (see picture 6). Always brake slowly and steady.

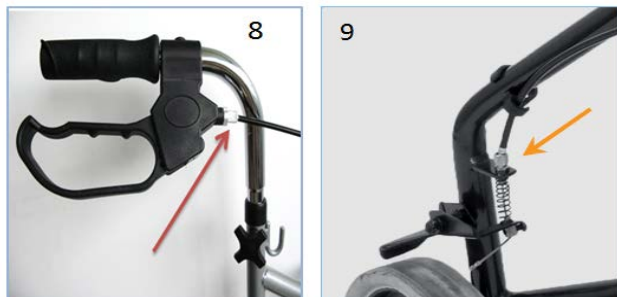
For parking the walking device press down the brakes until they click in the parking position (see picture 7).

For unlocking the brakes pull them upwards towards the handgrips until they are unlocked (both brakes are in a middle position).

Adjustment of brake

For desired braking strength adjust the brake either manually at the screw on the housing of the brake (see picture 8) or with the suitable tool (not included) on the brake wire (see picture 9).

Folding / Transport / Storage

To fold the TRI-walker for transporting, pull the black section back towards the handles. This will start the folding process. To allow the TRI-walker to stand upright during storage, do not fold the TRI-walker up fully. Leave a gap of at least 50mm (2 inches) between the rear tyres.

Safety instructions

Use only as a walking device!
Use only on even and solid surfaces.

Walking device must not be used for sit on or as a wheelchair! It must not be used for transport of persons or animals!

The effect of the brakes can be affected by abrasion of the wheels.
Therefore test regularly the brakes.

Maximum load: 130 kg

The use on escalator`s is expressly forbidden!

Use tray only for small, light, but not for pointed or hot objects. Max. load: 5.0 kg
Use bag and basket only for transport of objects and do not overload.
Max. load: each 5.0 kg

Do not expose the product to temperatures above 40°C for a long time, otherwise the steel pipes and the seat cover heat, which could lead to burns.

Cleaning / Maintenance

Clean regularly by hand with a damp cloth! Do not use any aggressive detergents!
Screws of walking device must be checked regularly and if necessary retightened.

Reprocessing

The product is suitable for reprocessing. Send it to an approved waste treatment centre.

Type Label

1. Product name
2. Article reference
3. Max.load
4. Batch number
5. Attention: See operating instructions

Warranty

We issue a warranty of 12 months from date of purchase. Subject to technical alterations. Errors and omissions excepted.

Technical Data

Dimensions	
Length over all with basket	600 mm
Width over all	610 mm
Width between handgrips	370 mm
Handgrip height	from 845 to 945 mm
Wheels diameter/Width	190 mm x30 mm
Dimensions folded (LxWxH)	650 x 250 x 855 mm
Weight	
without basket/tray	6,2 kg with
basket/tray	6,9 kg Weight
basket	0,5 kg Weight tray
0.2 kg	
Max. Load	130 kg
Material	Steel tube, high-quality coated

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