INTRODUCTION

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INTRODUCTION

COVID-19 is an acute respiratory disease, caused by infection via the SARS-CoV-2 virus. The SARS-CoV-2 virus belongs to the same family as the viruses which cause Severe Acute Respiratory Syndromes (SARS). The most common symptoms of COVID-19 are fever, fatigue and a dry cough. Certain patients may experience generalised pain, nasal congestion, rhinorrhoea (runny nose), a sore throat, diarrhoea, and loss of taste and smell. These symptoms are typically mild, and develop gradually. Some people may be infected by the virus, but present no symptoms. The majority of people (approx. 80%) recover from the virus without necessitating hospitalisation. The disease can be passed on from person to person by tiny droplets, which are transmitted from the nose or mouth when someone infected with SARS-CoV-2 coughs or simply exhales. Transmission by contact of contaminated hands touching the face is also frequent. Effectively, hands may be contaminated with the virus following contact with surfaces carrying the virus. The estimated period of incubation for the SARS-CoV-2 virus varies from one (1) to fourteen (14) days.

INTENDED USE

The Biosynex COVID-19 Ag Antigenic Self-testing Kit is a rapid, in vitro immunochromatography assay (also known as a lateral flow test) which allows for the qualitative detection of antigens of the nucleocapsid protein (N) of the SARS-CoV-2 virus, in samples of nasal swabs. It has been designed to assist in rapid diagnosis of infections of the SARS-CoV-2 virus. This test is a qualitative, rapid immunochromatography assay, which uses highly sensitive monoclonal antibodies to detect the nucleocapsid protein (N) of the SARS-CoV-2 virus. This self-testing kit allows you to identify whether you are, in fact, infected by the virus which causes COVID-19. This test is only to be used by adults, or under the supervision of an adult if the test is to be carried out on a child.

A negative result from the Biosynex COVID-19 Antigenic Self-Testing Kit for an asymptomatic individual does not wholly rule out a COVID-19 infection. If the individual presents symptoms suggestive of a COVID-19 infection, a negative result must be verified with a laboratory test.

KIT COMPOSITION**SAMPLING ADVICE**

- Wash hands before each test, in order to reduce the risk of contamination.
- For accurate results, do not use any samples which appear too viscous, or which visibly contain blood. You can blow your nose to remove any excess mucus before completing the test.

TEST LIMITS

This test does not determine the aetiology of the respiratory infection caused by micro-organisms other than the SARS-CoV-2 virus. The Biosynex COVID-19 Ag Antigenic Self-testing Kit can detect both the viable and the non-viable SARS-CoV-2 virus. The precision of the test depends on the quality of the SARS-CoV-2 virus. The accuracy of the test depends on the quality of the swab sample. False negative results may be given following poor sampling. Any failure to respect the test procedure may negatively impact the performance of the test and/or invalidate the test result. If the test is negative, yet clinical symptoms persist, it is advised that you carry out additional tests using other clinical methods. A negative result at no time rules out the presence of antigens of the SARS-CoV-2 virus in the sample, as they may be present but at a level inferior to the minimum detection level of the test, or if the sample has been collected incorrectly.

A negative result does not rule out infection by the SARS-CoV-2 virus, particularly in people who have come into contact with the virus. Follow-up tests with molecular diagnostics should be scheduled to rule out infection in these people.

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HANDLING INSTRUCTIONS

Keep the self-testing kit and its components out of reach of children; the diluent may pose a significant risk if ingested.

The used test cassette, as well as all components of the test, can be disposed of along with household waste, in a securely closed bag.

All components included in this kit are intended solely for use for this test. Do not re-use this test and/or any of its components.

This test must be carried out at room temperature (15 °C – 30 °C) and in an environment free from excessive humidity.

This is a single-use test.

This test must be used immediately after collecting the sample.

The extraction diluent contains a solution containing a preservative (0,05% sodium azide). If the solution comes in contact with your skin, wash with soap and rinse thoroughly. If the solution comes in contact with your eyes, flush with large volumes of water.

Solutions containing sodium azide may react explosively with lead or copper pipes. Use large volumes of water to thoroughly rinse waste disposed of in a sink.

Do not use the cassette if it has been removed from its sealed packaging for over an hour.

Do not use the cassette if its packaging is damaged.

STORAGE

This test must not be frozen, and must be stored in a dry place between 2 °C and 30 °C.

The test cassette must stay in the sealed sachet until it is about to be used.

Do not use the self-testing kit after the expiry date provided on the aluminium sachet, and on the outside of the cardboard box.

CROSS REACTIVITY

Cross-reactivity was observed with samples positive for the SARS-CoV pathogen.

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Please read the notice in its entirety.
Have a watch or timer to hand.

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Wash your hands before handling the kit to avoid any potential contamination.

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Tear off the top of the pouch, following the indent. Remove the cassette. Use the test within the hour.

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Remove the seal of the tube containing the diluent and place it in the cut-out hole in the box.

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Roll the swab 5 times along the mucous membrane inside the nose to ensure that the mucus and cells are collected. The swab must touch the inner walls of the nostril.

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Using the same swab, repeat this process in the other nostril. Remove the swab from the nasal cavity.

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Insert the swab into the tube and rotate it at least 6 times while pressing the end of the swab against the bottom of the extraction tube.

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Leave the swab in the extraction tube for 1 minute.

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Extract as much liquid as possible from the swab by "centrifuging" it using the edge of the tube. To do this, press several times on the edge of the tube. Once done, remove the swab from the tube and dispose of it.

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Attach the dropper tip onto the tube.

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Using the dropper, slowly add 4 drops of solution into well S.

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Start the timer, or note down the time of deposit. Read the results after 15 minutes. Do not read after 20 minutes.

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Place the red stopper over the dropper test after discarding the tube into the bin, in order to avoid any potential contamination.

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Refer to the "Interpretation" section for details on test reading.

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