
| Art.-Nr. | Description Bezeichnung | Height Hauteur Höhe | Width Largeur Breite | Depth/Profondeur Tiefe | Footprint/Dimensions Fußabdruck | | |
|----------|----------------------------|---------------------------|----------------------------|---------------------------|------------------------------------|--------|--------|
| 550806 | Accès o bain | 105 cm | 69 cm | 57.5 cm | 34x37 cm | 5.9 kg | 150 kg |

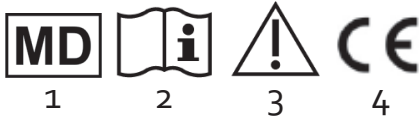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EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use in extreme temperature (above 38°C or 100°F or below 0°C or 32°F).
- ♥ Check carefully all parts of this item (screws, tubes, tips, grips...).
- ♥ Check regularly if all screws are tightened.
- ♥ Use only as a safety frame and in proper way : don't use as a lever, don't stand up on the safety frame.
- ♥ ALWAYS observe the weight limit on the labeling of your product.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After unfolding or assembling the safety frame, make sure that it is securely locked in OPEN position and level to the ground before use.
- ♥ DO NOT hang anything on the safety frame.
- ♥ ALWAYS test to see that the safety frame and attachments are properly and securely locked in place before use.
- ♥ In case you didn't use your safety frame for a long time, let it checked by the technical staff, before use.
- ♥ Do not store outdoor, the safety frame must be used at home.
- ♥ Check the safety frame before use, in case of doubt ask your physician or provider.
- ♥ Use only the safety frame on a flat and solid ground.
- ♥ Do not lean out of the safety frame to avoid losing balance.
- ♥ Any serious incident that has occurred in relation to the device shall be reported to the manufacturer and the competent authority of the Member State in which the user and/or the patient is established.

2. Intended use, indications and contraindications

Intended use:

The Herdegen Bath access kit has been designed to provide safety, comfort and independence for people who have mobility problems to access the bathtub. It is easy to install and remove without any tools.

Indications:

- Severe impaired mobility
- Temporary, durable, partial or total loss of strength in the lower limbs, balance disorders or posture problems

Contraindications:

This product is contraindicated in case of:

- cognitive disorder, Dementia
- severe loss of balance
- muscular insufficiency of upper extremities
- coordination disorder.

3. Installation

Only specialists can adjust the product and teach you how to use it. Ask the technical staff how to use this product and make sure that you understood clearly their explanation. In case of doubt, don't hesitate to ask again.

- 1-Take all the elements (see: (C)) out of the packaging and make sure that none of them has been damaged. In case of doubt, do not mount the product.
- 2-Insert the base [1] into the footstep [3] (see: (A)). The footstep can be placed on the left or on the right.
- 3-Place the locking system [2] on the bathtub rim by sliding it along the base [1] (see: (B)). Beware! Risk of injury! Ensure that the base is stable and that the locking system is placed as close as possible to the bathtub rim. Do not place the locking system against the bathtub so as not to scratch it.
- 4-Turn the locking system [2] (see: (D)) in order to fix the product on the bathtub rim.
- 5-Ensure stability before use.

4. Cleaning

The Bath access kit has to be cleaned regularly with a mild cleaning agent and a soft cloth.

5. Materials

The product is made up of steel tubing covered by epoxy painting, and plastic armrests. The bath stool is made of steel and wooden slats.

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible. Period of use : 2 years.