



INSTRUCTIONS FOR USE

WARNINGS:

- The device is provided sterile.
- Do not use if the packaging does not show intact.
- Use immediately after opening.
- The device should only be used by qualified medical personnel.
- The product is disposable. Do not use after use.

INDICATIONS FOR USE

The device is intended for the making of amniorrhexis ie those cases where it is necessary to cause an artificial rupture of membranes. The amniorrhexis is practiced during labor (it would be better not to perform it before the dilation of the cervix has reached at least 4-5 cm) to facilitate the start of the expulsion of childbirth period.

HOW TO USE

Introducing the Hospital's Break into the vagina, guiding it into place with the help of the index and middle fingers of the other hand. Breaking the membranes exercising with the distal end of the device a slight upward pressure. Slowly drain the amniotic fluid to avoid favoring risk of prolapse of the umbilical cord or placental abruption. When the membranes are set against the presenting part, it is good to use a vaginal speculum or an amnioscope performing amniorrhexis under visual control to be sure not to cause injury to the fetus or the mother

PRESERVATION / STORAGE

Store in a dry place. Do not expose to temperatures exceeding 50 ° C.

REMOVAL AND DISPOSAL AFTER USE

The product is disposable so it is not reusable, it can be burnt together with other waste, observing the provisions of the Local Authorities. L 'burning does not from place to the formation of toxic residues.

