W05

Instrucciones / Instructions / Instruções
Instructions / Anweisungen / Istruzioni /
Instruccions / Argibideak / Instruccions / Instruccions

W05

Componentes / Components / Componentes / Composants
/ Komponenten / Componenti / Components / Osagaiak/
Compoñentes / Components

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100% Polyester

Keep your purchase receipt to be able to exercise your rights as a consumer. The company reserves the right to make changes to the product with respect to what is described in this instruction manual.

Malfunction caused by natural disasters, maintenance by non-original factory, self-modification, is not applicable to warranty.

This product is used as a walking aid, not suitable for people who cannot walk independently.

For people with vision problems, dyslexia, or cognitive disabilities, share the instructions with family members.

Before using, please make sure all functions work well. Stop using it if the cart is found to be damaged or malfunctions and contact your authorized dealer.

This rollator is intended to be used indoors and by users with a maximum weight of 136 kg.

The maximum load weight in the basket is 5 kg.

Be careful and use the brake system when you walk on the ramp.

Do not store in a place with direct sunlight and heat/moisture, in order to avoid malfunction or deformation.

Do not use it on a ramp or a place with a difference in level in rainy days or slippery ground.

Incorrect use can lead to hazardous situation – Do not use the products to transport a person

Please, check Warranty Terms & Conditions at www.playcare.es

Washing instructions:

Read the washing label carefully.

Machine wash warm, gentle cycle. Dry line.

Do not use solvents or bleach that can damage the materials.

The metal and plastic parts can be washed by hand with lukewarm water and neutral soap. Do not use aggressive substances.

Carefully dry the cart frame after contact with water (or rain) to avoid moisture.

Do not modify or disassemble the cart frame to avoid damage or malfunction.

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Imported by: PLAY S.A.