English

Introduction

These guidelines give the correct methods for using liquid (water) re-usable FILTRANIOS PS1000 filter capsules supplied by Laboratoire ANIOS. If you have any queries, our process filtration specialists will be pleased to discuss your particular filtration requirements or answer any questions you may have. We may also be contacted at the addresses given on the reverse of this document or through our worldwide network of subsidiary companies and distributors.

Performance

The re-usable FILTRANIOS PS1000 filter capsules are dedicated to achieve a Q.2.1 water quality designated “bacteriological controlled water” by the technical guide “water in health establishments” edited by the French Health Ministry. The re-usable FILTRANIOS PS1000 filter capsules have the marking CE medical device IIB Class.

1. Storage

- 1.1 Store the capsules in a clean, dry environment and avoid placing heavy objects on the top of the capsules and/or packaging. The capsules should not be exposed to temperatures below 5°C (41°F) or above 40°C (104°F) or to direct sunlight.
- 1.2 Keep the capsule in its sealed polyethylene bag until it is time to install it.
- 1.3 The shelf-life for re-usable FILTRANIOS PS1000 filter capsules is 36 months. If a bag is opened or punctured the capsule must immediately be used.

2. Installation

2.1 First Installation

- 2.1.1 Remove all packaging.
- 2.1.2 Tick or clip the first number on the card.
- 2.1.3 IMPORTANT - unloosen the drainage cap.
- 2.1.4 Place the filter into a sterilisation bag and sterilise in an autoclave, according to manufacturers recommendations in section 4.
- 2.1.5 Ensure that the inlet connection of the capsule matches the tap quick connect coupling.
- 2.1.6 Drain the tap.
- 2.1.7 Open the sterilisation bag.
- 2.1.8 Keep the filter in the bag.
- 2.1.9 Push the filter into the tap's quick connect coupling and dispose of the bag.

2.2 Usage protocol

- Beginning the operating day*
- 2.2.1 Drain the tap
 - 2.2.2 Open the sterilisation bag.
 - 2.2.3 Keep the filter in the bag.
 - 2.2.4 Push the filter into the tap's quick connect coupling and dispose of the bag.
 - 2.2.5 Filter ready for one operating day
- End of operating day*
- 2.2.6 After use, remove the filter from the tap's quick connect coupling.
 - 2.2.7 Unscrew the drainage cap. Turn the filter and allow residual water flow out.
 - 2.2.8 Replace the purge cap but DO NOT tighten.
 - 2.2.9 Tick or clip the next number on the card.
 - 2.2.10 Place the filter into a sterilisation bag and sterilise in an autoclave, according to manufacturers recommendations in section 4.
IMPORTANT: The product must be sterilised every 24 hours.

3. Operation

Re-usable FILTRANIOS filter capsules should not be subjected to excessive hydraulic shock and should never be reverse pressurised from the downstream to the upstream side (inside to out). Open the tap by using the appropriate system (photoelectric cell, lever,...), then close the tap after use. When the filter reaches it maximum sterilisation number, remove the capsule and destroy it. Re-usable use FILTRANIOS capsules are shown to be blocked when the differential pressure across the filter has significantly increased and/or the flow of liquid through them is reduced to an unacceptable level. In that case, please contact Anios Laboratories for optimisation of capsule usage.

4. Sterilisation

The re-usable FILTRANIOS PS1000 filter capsules have been validated for a limited number of sterilisation, depending on the autoclave cycle:

60 cycles - 18 minutes @ 134°C
90 cycles - 20 minutes @ 120°C

5. After Usage

After use, the capsules are considered as medical waste, comparable to non-dangerous waste (as packaging), and without direct risk for the health of people or the environment.

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