SECUBACK I

SECUBACK II

SECUBACK III

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Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Do not store outside and do not let under the sun (Beware: don't let the product in a car in case of high outside temperature !!).
- ♥ No modification or repair must be made on this device.
- ♥ Follow the advice of the professional who prescribed or supplied the product.
- ♥ This device was prescribed by a professional and fits you, it should not be used by another person.
- ♥ Consult your professional if discomfort.

2. Indication

SECUBACK belts are meant to protect the person who is seated in the wheelchair

to prevent them from sliding or falling out of the wheelchair.

3. Description and Fitting instructions

SECUBACK belts are suitable for wheelchair or commode wheelchair users.

Ventral belt SECUBACK I

This padded belt is made of thick foam padding for comfort. It is meant to be used in a wheelchair to prevent users from sliding forward. The hook and loop closure, located at the back of the belt, prevents users from removing the belt.

Installation:

Wrap the belt around the patient's stomach and the wheelchair's backrest. Use the hook and loop closure to secure the patient and place it at the back of the wheelchair to ensure that the patient does not remove it on its own. Height: 20cm Size: 130cm or 150cm

Pelvic vest SECUBACK II

SECUBACK II features suspenders to firmly secure the patient and prevent chest collapse.

Adaptable to any wheelchair.

Installation:

Wrap the belt around the patient's stomach, then cross the suspenders in his back and use the hook and loop closure to fix them on the strap.

Size: 130cm or 150cm.

Pelvic belt SECUBACK III

SECUBACK III ensures a comfortable sitting position and prevents patients from sliding forward.

Reduces the risk of shear stress and artery compression.

Installation:

Wrap the belt around the patient's stomach, then slide the strap under the wheelchair's seat so as to fix it to the belt with the hook and loop closure.

Size: 130cm or 150cm.

4. Washing instructions

Wash with soapy water not more than 40°C. Rinse thoroughly with lukewarm water. Squeeze out excess water. Dry flat and away from any source of heat. Do not use detergent or bleach. Do not iron.

5. Composition

See product label. Laminated fabric : 100% polyester.

Foam : 100% polyuréthane

6. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

7. Technical specifications

This product has a 2-year warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible and CE certified.

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